External Spray E-type Contra Angle Handpiece

Operation Manual

 The EU directive 93/42/EEC was applied in the design and production of this medical device.

☐ **Ultimate BA101**

REF 9792837

☐ **Ultimate BA105**

REF 9792838

☐ **Ultimate BA105T**

REF 9792922

Thermo-Disinfectable

B.A. INTERNATIONAL LTD.

Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.

Tel: +44 1604 777 700

Rev.2016/08

AUTOCLAVABLE

Caution before use

- The BA Brand Ultimate external spray contra angle handpiece is intended for use by dental professionals only.
- Read this Operation Manual carefully before using the contra angle handpiece. Be sure to follow all the operating instructions, including all cautions and warnings.
- Keep this Operation Manual for future reference.

Specifications

| | BA101 | BA105 | BA105T |
|----------------------------------|---------------------|-----------------|--------------------------------------|
| Applicable Bur | 2.35 mm CA bur | 2.35 mm CA bur | 2.35 mm CA bur |
| Insertion/Removal of Bur | Standard Latch Type | Pushbutton Type | Pushbutton Type |
| Allowable Maximum Speed | 30,000 rpm | 40,000 rpm | 40,000 rpm |
| Nose Configuration Sterilization | Autoclavable | Autoclavable | Autoclavable Thermo-Disinfectable |

Note : The maximum speed may depend directly on the type of tubing and type of motor.

Removal/Insertion of Bur from/into Contra Angle Handpiece

BA101

- Swing latch plate out. Remove the existing bur blank.
- Insert a bur into chuck hole all the way until the end of bur shank touches the back of the chuck.
- Swing the latch plate back until it is fixed with a snap at the centre position of the collar of the contra angle head.

BA105 and BA105T

- Push the pushbutton with thumb to open chuck, and then remove the existing bur blank.
- Push the pushbutton with thumb, and then insert a bur into the chuck hole.
- Release the pushbutton with thumb when the end of the bur shank touches the end of chuck.

Caution

- Before use, make sure that a bur is securely locked, pulling it gently with fingers. Make a trial run.
- Never use a non-ISO standard bur for the contra angle handpiece.
- Never use a bur with the bent shank or a damaged bur for the contra angle handpiece. Such a bur may not only injure the patient but critically damage the contra angle handpiece.

Installation of Contra Angle Handpiece onto Motor

- Make sure that the motor onto which the contra angle handpiece is attached is an ISO E-type.
- Insert the E-type fitting of the motor into the back-end of the contra angle handpiece.
- Make sure that the contra angle handpiece is securely installed onto the motor, by pulling it gently with fingers, and then make a trial run before use.

Warning

- If the contra angle handpiece cannot be smoothly attached/removed onto/from the motor or if the contra angle handpiece rattles after installation, never operate the contra angle handpiece.
- Whenever abnormality in performance, sound level, heat level, function, etc. is sensed, stop using the contra angle handpiece immediately and consult **BA International Ltd. or an accredited BA certified Service Centre.**
- Never install/remove the contra handpiece onto/from the motor while the motor is in operation.


Caution

- Never operate the contra angle handpiece by installing it onto a motor which runs beyond the allowable maximum speed, as the excess speed may cause critical damage to the internal mechanism.
- Never dismantle or alter the contra angle handpiece.

Lubrication

- Use **BA ULTIMATE SPRAY.**
- The contra angle handpiece must be lubricated once a day or before autoclaving using the following procedures:
 - Remove contra angle handpiece from motor.
 - Remove the bur.
 - Wipe off the saliva, dirt, etc. stuck to the surface of the contra angle handpiece with an alcohol-soaked soft tissue or cotton swab.
 - Keep the contra angle handpiece in a horizontal position and inject BA ULTIMATE SPRAY for three seconds through the back-end of the contra angle handpiece using the appropriate (E-Type) nozzle.
 - Install the contra angle handpiece onto the motor after the spray injection.
 - Make a trial run for five seconds.
 - Wipe down the excess lubricant on the surface of the contra angle handpiece with a soft tissue.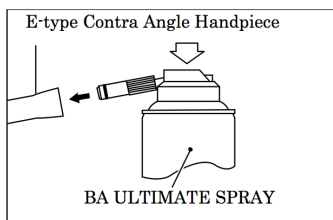

Caution

- Never use a wire brush or disinfectants for surface cleaning.
- Never immerse the contra angle handpiece in an ultrasonic cleaner or disinfectant solution.
- Lack of lubrication after use or before autoclaving may cause premature failure of the contra angle handpiece.

Sterilization

An autoclave is recommended for sterilization.

- Before sterilization, clean the surface of the contra angle handpiece.
- Lubricate the contra angle handpiece, and then make a trial run.
- Remove the contra angle handpiece from the motor.
- Put the contra angle handpiece into a sterilization pouch when using a vacuum type autoclave. Do not combine it with other instruments in the same pouch.
- Autoclave the contra angle handpiece at 121 degrees C for 20 minutes or 134 degrees C for 15 minutes.

- BA handpiece with this symbol can be cleaned and disinfected in a thermo washer disinfecter.
- Follow manufacturer's operating instructions including caution, warning, etc. in use.

Warning

- Never use dry heat sterilization, cold sterilization, etc.
- Never autoclave the contra angle handpiece together with others in a sterilization pouch.

Installation of Contra Angle Handpiece onto Motor


Removal

- Turn union nut counterclockwise with fingers.
- Pull the head out.

Connection

- Insert transmission gear into the head, and then rotate it with fingers so that the gear of the transmission gear engages with that of the head.
- Engage crown of head with that of shank and then turn the union nut clockwise

Note: In case of latch type, remove the head after swinging the latch out.

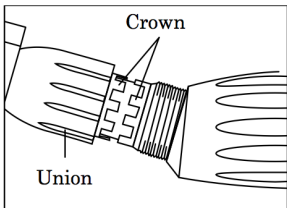Service and Repair

Important

This contra angle handpiece is a result of precision engineering and cannot be dismantled. For service and repair, always send it to either BA International Ltd. or an accredited BA certified service centre without dismantling it.

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