BA182050
9796027

BA Optima E+ **Cordless Endodontic Unit**

BAE370R

Rev.2019/03

INSTRUCTION FOR USE

BA Optima E+ Cordless Endodontic Unit

Read this operation manual carefully before use for operation instructions, care and maintenance. Keep this operation manual for future reference.

- 1. WARNING 4
- 2. MACHINE CLASSIFICATION 6
- 3. PRODUCT FEATURES 6
- 4. PRODUCT SPECIFICATION 7
- 5. COMPONENTS 7
- 6. PART NAME OF PRODUCT 7
- 7. LCD AND OPERATION PANEL 8
- 8. OPERATION PROCESS 10
- 9. ERROR AND REMEDY 12
- 10. MAINTENCE OF THE PRODUCT 12
- 11. TROUBLE SHOOTING 14
- 12. WARRANTY 14
- 13. SYMBOL 15

1. WARNING

- Please read the warnings before operation.
- Warnings are to prevent injury and damage to user and patient. These are all for your safety, please follow the instructions.

• Class II equipment

To identify equipment meeting the safety requirements specified for Class II equipment according to IEC 60536.

Note - The position of the double-square symbol has been placed such that it is obvious that the symbol is part of the technical information and can in no way be confused with the manufacturer's name or other identifications.

WARNING

- Only rechargeable batteries are allowed to be used. Do not use manganese and alkaline batteries. If mishandled, this may cause leakage or ruptures.
- Do not touch the AC power plug with wet hands. This is a shock hazard.
- Do not drop any kind of liquid or water on the motor handpiece and adapter, as a short circuit may cause fire, electric shock or corrosion of the product.
- Unauthorized modification and dismantling is prohibited.
- Do not drop the motor handpiece or adapter. Install the adapter on a flat and safe surface.
- If the product is not to be used for long periods, please remove the battery. There is possibility of leakage.
- Please use the same kind of batteries. Otherwise, there is risk of leakage of dissimilar chemicals that may react.
- Please use the designated batteries and charger for the product.
- Do not disassemble or change the motor.

NOTICE

- The motor handpiece is designed to be used with Nickel mercury batteries (rechargeable). It is possible to use Ni-Cd battery but the charging time and usage period may be shortened due to the charging currents being different.
- The motor handpiece will draw a small electric power during power off. The rechargeable battery will discharge little by little when the product is not in use even after charging fully. The product is recommended to be charged before use.
- Machine is equipped with low voltage sensor. however, sometimes machine fails to perceive low voltage especially after a re-charge for

WARNING SYMBOLS

Warning Classification	Degree of Injury and Damages
 Prohibition	The Warning is to protect from loss of life and property
 Warning	The Caution is to protect from serious and minor injuries and possible product damage
Caution	Mandatory Notice to notify of safety measures.

- System damages may occur if the product is used near an electro-magnetic field.
- Do not use the product near of the flammable product or nitrous oxide.
- The product has circuit which can protect of file damage, but the file can be damaged by strong torque or resistance.
- Please do not leave the child alone with the product and keep attention to child, elderly and disabled person to be close to the product or not.
- Do not touch signal input, signal output, or other connectors, and the patient simultaneously.
- External equipment intended for connection to signal input, signal output or other connectors, shall comply with relevant IEC Standard (e.g., IEC60950 for IT equipment and IEC60601-1 or IEC60601-1-1 series for medical electrical equipment) If in doubt, contact qualified technician or local representative.

- a few minutes, which is normal with this type of battery.
- Please charge the battery (which is rechargeable) after use, as often as possible. Periodically, discharge, and charge, as short usage periods can shorten battery life (due to the memory effect) In this case, repetition of charge and discharge of fully used battery can make it better condition.
- If the product is not in use for long period, please take out the battery.

CAUTION

- Please consider the safety of patient first and take care during the product operating.
- Only qualified person in dental surgery can be allowed to use the product
- Please purchase the battery recommended by manufacturer and take care of the usage manual of battery manufacturer before use it for the product.
- Do not use the machine with the file which is bended, damaged, deformed and/or not in ISO standard, otherwise, it might cause injury by some possible fragments.
- Do not place or maintain the battery in the place in high temperature such as the place exposed to direct sunlight, inside of a car in high temperature or beside of stove and fire and so on.
- Please take care of shaving, vibration, noise and heat of the product before use and test by rotating naturally. Stop using the product if condition of the product feels abnormal and contact a representative.
- Keep bur or file clean, otherwise dust might cause poor chucking power and it may cause vibrations during operation. Users are responsible for operation, maintenance and inspection.
- Applications of machine is restricted by this manual and not recommended applications by a manufacturer should be avoided.
- An arbitrary use might cause users' injury and loss of property and the a manufacturer shall not be responsible for the results which are made by the operation which is not mentioned in this manual.
- Keep the speed of the usage tool as recommend by tool manufacturer. Exceeding the speed limitation might cause an accident.
- Contact dealer when display shows wrong message, it might cause an accident.
- Only recommended product and consumable parts by manufacturer have to be used with the product, otherwise, it may cause an the accident.
- Operating Condition
 - 1) Temperature: 10°C ~ 30°C
 - 2) Humidity: 10% ~ 80%
 - 3) Atmospheric Pressure: 700 ~ 1,060hpa
- The temperature of the handpiece goes up to max. 41°C when atmosphere temperature is 30°C.
- Before changing the head or file, turn off the power of the motor handpiece. Changing it during power on may cause unintended rotation by accidental activation of the ON/OFF button.
- When inserting the motor handpiece into the battery charger, position the handpiece correctly. Pushing it into the charger forcibly in the incorrect direction may cause damage.
- Take care not to put needle, pin into the charging terminal of the battery charger.
- Place the motor handpiece into the charger after clean the liquid or dust off. If the charging terminal is in dirty, it can be rusted and cannot be charged properly.

- Do not lubricate the motor handpiece.
- Do not heat sterilize the motor handpiece. Do not autoclave the motor handpiece.

ELECTRIC SAFETY

In case of some faults in Electric Cord, Plug, and Connector, do not use machine, otherwise it might cause Fire and Electric Shock. Do not pull out the Plug with wet hand. Whenever the Plug to be pulled out, to be sure to grasp the Plug body. Do not bend Electric Cord excessively. Do not put heavy material onto Electric Cord.

- Never fail to use Outlet which is equipped with earth connection and please be instructed the way how to install the earth connection from manufacturer or technicians.
- Do not put several Plugs into one Outlet though multi-plug, which might cause Fire and Electric Shock.
- Before use, please check if the control unit has set the gear ratio to match with the gear ratio in the contra angle.
- If motor operating speed is too fast or stop during operation, contact dealer.
- Take care not to drop or damage the motor and contra angle handpiece. Do not operate the product if product is not operating properly after dropping on the floor or in water, instead contact the Head office or Dealer. That case may cause bur vibration, excessive heat and damage to the ball bearings.

DISPOSAL

- When disposal of Control units, be sure to keep regulations which stipulate disposal of industrial waste.
- When disposal of Motor and Handpiece, be sure to keep regulations which stipulate disposal of medical waste.
- Used and finished battery is recommended to be returned to supplier. The product use battery which is composed of Nickel Metal Hydride, rechargeable, however, some countries stipulate it to be disposed as special waste.

INTENDED USE

BAE370R, BAE07, BAE05 and BAE03 are for indicated for use by dentists in standard endodontic procedures using rotary files and rotary endodontic drills (Gates-Glidden)

2. MACHINE CLASSIFICATION

- Type and Degree of protection against electric shock:

Class II/Type BF

Degree of protection against water (vertically falling drops) stipulated in IEC 60529 : **IPX 0.**

Machine should not be used at the place close to flammable Anesthetics and Oxygen.

Operation Mode: intermittent load in consecutive operation

Loading Time: max. 3 minutes

Resting Time: min. 10 minutes.

3. PRODUCT FEATURES

- Simple interface design for easy control
- Freedom of activity during operation by wireless handpiece
- Ergonomic and compact body

<Endo mode>

- RPM : (10:1) 200~900RPM (900rpm±10%)
(16:1) 100~600RPM (600rpm±10%)
(20:1) 100~600RPM (600rpm±10%)
- Torque : (10:1) 1~2N-cm (16:1) (20:1) 1~4N-Cm
- Gear Ratio : 10:1, 16:1, 20:1,
- Auto Reverse Mode : Level 1, 2

<Program function>

- 5 memory programs of (Gear Ratio, Torque, RPM, Direction mode) are available to set
- Programs no. 6 and 7 are reciprocating modes and the user can set desired angle for forward or reverse rotation

<Setting torque limit function>

- If actual load on the file is higher than preset torque, it stops working automatically.

<Auto reverse mode>

- Users can choose Auto-mode according to operation condition
 - Level 1: Once actual torque is over the preset torque, the file move in Reverse rotation. After removing the load, it stops.
 - Level 2: Once actual torque is over the preset torque, after auto-reverse and load released, it turns forward-rotation again.

<Auto-OFF function>

- Power is off with "Beep" sound after 4 minutes without operation. (During the motor operation even without use, power does not switch off automatically)

Machine has been confirmed that it complies with the rules of medical machinery IEC 60601-1 : 2007 which provides reasonable protections against harmful interference in typical medical installations.

Machine can radiate radio frequency energy. If not installed or used in accordance with the instructions, it might cause harmful interference possibly to other devices in nearby, however, it is not assured that it will occur to a particulate installations.

If machine cause harmful interference to other devices and that interference is determined by their ON/OFF, users are encouraged to try to correct the interference by one or more of the following measures:

- Machine needs to be used by a medical specialist or qualified person.
- Do not use machine at the place too close to other devices.
- Consult with manufacturer or qualified service technicians for the electronic safety information of the machine.

4. PRODUCT SPECIFICATION


Charger	Motor Handpiece	Angle Handpiece		
BA182511	BA182510	BAE07 BA680057 ACL(B)-41EP	BAE05 BA680055 ACL(B)-42EP	BAE03 BA680053 ACL(B)-46EP
Input : AC100-240V, 50/60HZ, Max0.1A	Rated Voltage : DC 2.4V±20%	Gear Ratio : 20:1	Gear Ratio : 16:1	Gear Ratio : 10:1
Output : DC 4.5V	Battery : 1.2V X 2EA= 2.4V	Dimensions : Ø16.7 X 65.8mm	Dimensions : Ø16.7 X 65.8mm	Dimensions : Ø16.7 X 65.8mm
Charging time : 2.5hours		Weight : 34.8g	Weight : 34.8g	Weight : 34.8g

5. COMPONENTS

- Angle Handpiece (Contra Angle)
- Motor Handpiece
- Charging Station
- Power cord


6. PART NAME OF PRODUCT

- 1- Angle Handpiece (Contra angle)
- 2- Motor Handpiece
- 3- Motor ON/OFF button
- 4- Charging light
- 5- LCD operation panel
- 6- Operation buttons
- 7- Power button
- 8- Charging Station
- 9- Charger ON/OFF button

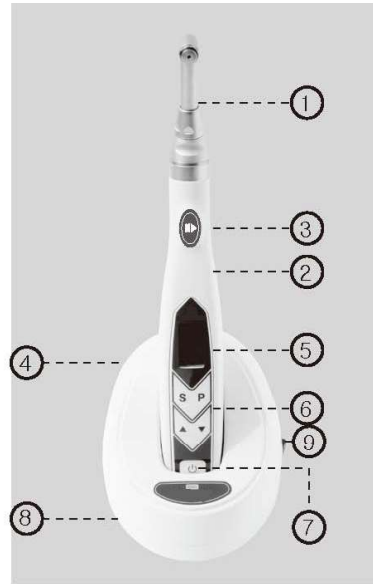7. LCD AND OPERATION PANEL

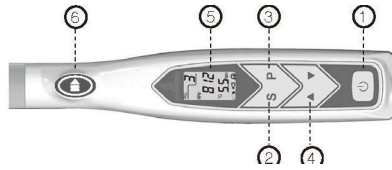1 Power Button

Push the power button until the power is on and the LCD panel lights. Push the power button until turn off over the 1 second.

2 Select-S Button

This button is to select the settings for RPM, torque, gear ratio and auto reverse mode in programs 1-5. In programs 6-7 (reciprocation mode), this button is used to select the settings for forward rotation angle and reverse rotation angle.

3 Program-P Button

User can select a program from no. 1 to 7 and also use this button to save settings in the memory. Up to 7 user defined settings can be saved (programs 6 and 7 are reciprocation modes). (It can be the button for Program selection and Memory button as well.)

4 Up/Down button

The user can set needed RPM, torque, gear ratio (programs 1-5) and reciprocal angle (programs 6 & 7) using the Up/Down buttons. In programs 6 & 7 (reciprocation mode), the Up button is to increase, and the Down button to decrease the reciprocal angle.

5 LCD Panel

Display shows Program, Speed, Torque, Auto reverse mode, Battery and Beep sound mode on the screen.

6 Motor ON/OFF Button

Push the Button, then motor runs and stops.

Auto Reverse: If the load is removed after auto reverse rotation, it returns to the normal rotation again.

No display

Auto Reverse Off: Auto reverse rotation is not activated.

Battery Symbol: shows 3 Steps for charging amount

Beep Sound Symbol (On/Off): Beep sound function for each operating condition

Program: shows current program and control from Step 1 to 5

Torque: controls torque, shows current torque limitation (Measurement: N.cm)

Speed: controls rotation speed

Gear Ratio: shows current Gear Ratio and possible setting Gear Ratios are 10:1, 16:1, 20:1 ; RPM and Torque are set by Gear ratio so check the setting condition of Gear ratio before operating.

LCD panel

Torque setting screen

Gear ration setting screen

Reciprocation forward rotation angle setting screen

Reciprocation reverse rotation angle setting screen

8. OPERATION PROCESS

Operation Method

1. How to charge

- Plug the Power cord jack into the inlet placed back side of Charging Station.
- Insert the Power cord and plug in.
- Turn on the Power switch and check if the power lamp is on and Beep sound is ringing Place the motor handpiece into the Charging Station. Charging Lamp is on and starts to be charged.
- Charging lamp lights Green when the battery is fully charged and lights Orange while it charged. The Orange light is blinking when it has short-circuited or battery needs to be changed

Warning

- Never use the Charging Station for other purpose except the motor handpiece
- Charging time is normally 2.5 hours, but it depends on battery usage conditions, battery freshness, ambient temperature, etc. Older batteries are especially prone to significantly shorter charging and operating times.
- Batteries may warm up during charging, but it is not a failure.
- The battery is defective if the Orange light is blinking during charging. Charge the battery after replacing with a new one.

2. Changing Contra Angle

Contra angle can be connected to Motor handpiece at any position. Align Contra angle positioning pins with the positioning slots of Motor handpiece and insert the head until "Click" Pull it out axially for removing.

Warning

Check if the Contra angle head connected properly to the motor handpiece. Otherwise, the motor handpiece is not operating properly.

3. Changing File

- Insert the File to the chuck
- Push it into until "Click" sound.
- File removing: Press Push Button and pull the file out

Notice

- Charging Lamp may change from Orange to Green light quicker than normal when charging, depending to the battery charge level.
- To keep the motor handpiece putting into Charging Station for a long time, it may not be charged properly.
- If the Orange lamp is blinking due to short circuit, contact dealer or manufacturer.
- Charge the motor handpiece during its inactive time.

Warning

- Turn off the power to remove or connect the contra angle head.
- Pull the file slightly after inserting, to check if the file has been inserted properly.
- Clean the shank of the file prior to insertion.

Allowing dirt to enter the chuck could cause loss of concentricity and deterioration of gripping force.

4. How to operate

- Turn on the power by pressing down the Power Button.
- Press the Program button until the program number suitable for the file to be used is displayed. (Press down the program button until you hear a 'beep' sound to save the current settings to memory). Program from No.1 to 5 is Normal mode, Program from No.6 to 7 is Reciprocating mode.
- The Select button is to change the set value of auto-rotation mode, torque, gear ratio or reciprocal angle to match the file being used. (for reciprocal angle: press Select for forward rotation, then press Select again for reverse rotation). To change values, use the Up/Down button; then save the set value by pressing and holding the Program button.
- Operate the product by pressing down On/Off Button.
- After operation, put the motor handpiece into the stand; turn off by pressing down Power Button.

Warning

- Due to Automatic Power Off function, after 4 min. without On/Off Button operation, the unit turns off automatically. However if the device is operated, the Power will not turn off automatically.

5. Program

- There are 5 normal programs available and it can set the initial and setting value-RPM-Torque-Gear ratio. There are 2 reciprocating programs where the user can freely adjust the angle for forward rotation and reverse rotation.
- Find the needed program among set program.
- Control the button to adjust Auto rotation mode-RPM-Torque-Gear Ratio or reciprocal angle.
- Sounds "Beep" after program memorized by pressing down Program Button

Warning

- If load on the file reaches the preset torque, alarm sound changes. If load on the file is over the preset torque, three modes can be selected as below.
- The three Auto-rotation modes are available to be set by pressing down Select (S) button three times. Then press down the Up/Down button to select the mode.

6. Auto Reverse Function

- If the load reaches to torque limit value, it starts reverse with "Beep" sound. At this time, two auto reverse modes are available.

1 Auto Stop

* Motor handpiece move in Reverse movement. After removing the load, handpiece stops. To operate it for forward movement, press On/Off Button again.

2 Auto Reversing

* If the load is removed after auto reverse rotation, it returns to the normal rotation again.


Warning

- When the battery level is depleted i.e when the remaining charge indicator is low the actual load may be to high for the device to reach the preset torque limit value. In this case (in the case of operation by batteries), the auto reverse function will not be activated. When high torque is required, use this product in a condition where the batteries are not significantly coonsumed (As indicated by the battery indicator)
- If a load is continuously applied to the motor handpiece, it may automatically stop to prevent overheating. (The torque limit value indicates "OK" on the LCD panel). In this case, leave the motor handpiece for a while until it cools down. When the display returns to normal, the motor handpiece can be used.

9. ERROR AND REMEDY

ERROR MARK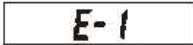

REMEDY

- During performing the operation, in the case that the actual load exceeds the torque value set, it will be automatically reversed. At that time, when it exceeds the set torque it will be automatically stopped and Error code (E-1) will be displayed on the screen.
- Remedy : Push Motor ON/OFF Button

10. MAINTENANCE OF THE PRODUCT

1. Cleaning

>> Angle Handpiece

- Rotate the head in a tap water about 10 seconds with maximum speed to remove the remains (blood, physiologic saline solution, etc.) and rinse.
- Do not put the tap water in the motor.
- Disconnect the angle handpiece to the motor handpiece.
- Remove contaminants on the angle handpiece using a nylon brush after soaking for 20 minutes in Enzymatic Detergent.
- Rinse the angle handpiece using the tap water for 3 minutes to remove Enzymatic Detergent.
- Wash the angle handpiece for 10 minutes in the ultrasonic cleaner containing Enzymatic Detergent.
- Rinse the angle handpiece for 3 minutes in the ultrasonic cleaner containing purified water repeatedly subjected 3 times. (Change the purified water each time.)
- Wipe using dust-absorbent after the product soaked in alcohol(70%) for 30 seconds.

>> Motor Handpiece

- Wipe the surface of motor handpiece with alcohol swab or paper towel after removing the dirt.
- Pay attention to not enter the water into the motor handpiece.

2. Sterilization

Sterilization product list

- Angle handpiece

Sterilization Method

- Dry the applicable product completely after cleaning.
- Put the applicable product in a sterile bag and seal tightly.
- Put the sterile bag in the sterilizer and sterilization for 4 minutes with 132 °C, and let it dry for 30 minutes.

Sterilization	Temperature	Time	Drying time
Moist heat sterilization	132 °C	4 minute	30 minute


3. Lubrication

Lubrication product list

- Angle handpiece

Lubrication Method

- Insert the spray nozzle to motor assembly part (the end part of angle) and spray lubricant 2~3 seconds.
- Insert the nozzle tip to head part and spray lubricant 2~3 seconds.

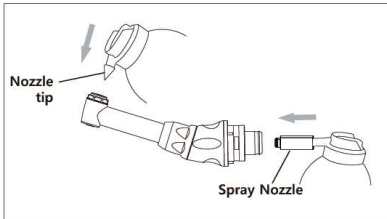Notice

- The nozzle tip and spray nozzle are not provided by manufacturer.
- If the continuous operation, please put the oil every an hour.
- When lubrication, please hold the handpiece firmly because the handpiece can go out by the spray pressure.

Warning

- Do not lubricate the motor section of the motor handpiece.
- To clean the motor handpiece, do not use any solvent such as benzene and/or a thinner.
- Before mounting the lubricated contra angle head to the motor handpiece, wipe off extra oil. Stand on its end or lean it in the proper position for gravity draining. Mount it after excess oil has been drained.
- Hold the contra angle head securely to prevent it from flying off by the pressure of the spray.
- Supply lubricant until it comes out of the handpiece head (for approx. 2 seconds.)
- Do not use a spray can upside down. In such a case, only spray gas comes out and oil is not sprayed.
- Do not clean with aldehydes for the plastic component. (Charging Station, Motor handpiece)

11. TROUBLE SHOOTING

When trouble is found, check the following before contacting dealer. If none of these are applicable or the trouble is not remedied even after action has been taken, contact dealer.

Problem	Cause	Solution
The power of motor handpiece is not turned on.	Battery has been discharged.	Charge the battery.
	Battery has not been inserted.	Insert the battery.
The temperature of motor handpiece is high.	The temperature of battery is high.	It is normal that the batteries become a little bit warm right after charging. If the batteries are hot while normal operating conditions, there may be an abnormality. Contact dealer.
The Charging Station does not work. (The CHARGE lamp does not light.)	The plug of the Charger is not inserted into the outlet, or there is no electricity in the outlet.	Check the connections.
	Internal fuse has burnt.	Contact dealer.
The motor handpiece does not rotate	Head of Contra angle was stuck.	Clean the head or change it.
	Bonding of motor gear was detached.	Contact dealer.

GENERAL MANDATORY

- * Clean inside of the product before sterilizing to avoid dirt and blood stuck inside.
- * Do not use the product in wrong way and be careful for safety.
- * Disassemble the product after stop the product completely.
- * Disassemble and/or repair only the case which mentioned in the manual.
- * Do not drop the handpiece, it may cause the damage of the product.
- * Use the product after inserting the contra angle into the motor handpiece end completely.
- * Check if it is normal condition for handpiece operation, especially for vibration, noise, temperature etc.
- * Contact Dealer/Manufacturer if problems occur.
- * The manufacturer does not have any responsibility for problems that occur after not following the manual.
- * Product design and specification can be changed for better quality without advance notification.
- * User has a responsibility for use and maintains the product properly.

12. WARRANTY

(Handpiece=1 year/Control Box=1 year)

- * The warranty does not cover the damages by user's misuse and normal wear and tear of spare parts.

Service Charge

In case of fault or malfunction through misuse of the user

- Using with wrong Input Voltage
- In case of user's misuse or improper use
- Dropping the unit during setting, moving or using
- Using consumable parts or accessories not recommended by manufacturer
- Malfunction after repairing at the places not recommended by manufacture

Other cases

- Fault by Act of God (fire, sea wind or water, flood etc.)
- Replacing due to damage by normal wear of consumable parts (Ball bearings, spindle case ass'y, or motor etc.)

Operating condition	Temperature (°C)	10-30
	Relative humidity (%)	10-80
	Atmospheric pressure (hPa)	700-1,060
Transport and Storage condition	Temperature (°C)	-20-60
	Relative humidity (%)	10-90
	Atmospheric pressure (hPa)	500-1,060

13. SYMBOL

Manufacturer

SN

Serial number

Autoclave

Type BF

This product should not be mixed with other commercial wastes for disposal.

Read the instructions

Read the instructions

General mandatory action sign

General warning sign

General prohibition sign

CE Marking

Class II equipment

① Power

② Select-S

③ Program-P Memory

④ Up/Down

⑥ Motor On/Off

MANUAL DE INSTRUCCIONES

BA Optima E+ Cordless Endodontic Unit

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BA International Ltd.
SAESHIN PRECISION CO., LTD.