SuPreme S5

Steam Cleaner Generator

MODEL: BA190030

Operating instructions
Modo de empleo
Mode d'emploi
Betriebsanleitung
Gebruikershandleiding
Istruzioni per l'uso

(manufacturer)

B.A. INTERNATIONAL LTD.
Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.
Rev.2016/01

Operating instructions

Steam Cleaner Generator

1. MANUAL USE

Read this manual through carefully before commissioning the machine, that is, the system. The purpose of this manual is to provide the user with all the information he needs to be able to use the machine properly and in the safest and most autonomous way possible.

It includes information about the technical aspect, operation, machine stops, maintenance, spare parts and safety.

Before anything is done on the machine both the operators and qualified technicians must read all the instructions given in this manual carefully.

If there are any doubts about the correct interpretation of the instructions please contact our office for the necessary clarifications.

This manual is an integral part of the machine and as such must be treated with care by the purchaser; it must be kept near the machine in a special place and, above all, protected from liquids and anything else that could ruin its legibility.

The manual must accompany the machine if it is sold to a new user.

It is forbidden for anyone to disclose, modify or use this manual for their own purposes.

2. SYMBOLS

Symbols highlighting situations requiring caution, practical tips and simple information have been used to help you read and quickly assimilate the contents of this manual.

Pay attention to the meaning of the symbols: they are intended to avoid repeating technical concepts or safety warnings, and should therefore be regarded as reminder. Consult the chart below if you are in doubt as to their meaning.

Dangerous-Attention


Disconnect power supply

Prohibition

3. MARKING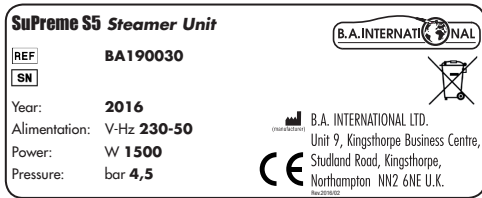

- The machine is made in compliance with the relevant Community Directives, applicable at the time it was placed on the market.
- Since this machine is not included in Enclosure 4 of the 2006/42/EC directive a self-declaration of EC conformity is issued. Conformity model “schedule A”.
- The plate on the machine must be similar to the one shown above and duly filled in.

4. SECURITY

- Do not fill in the boiler with any other substance than normal water, demineralised or distilled water.

- Point the gun so the steam jet points only in the direction of the job you are performing. Do not point it in any other direction.
- Turn off the machine when you have finished using it.
- To prevent accident the steamer should be turned off after working hours, during breaks and secured against unauthorized access.

- Do not exceed maximum /minimum water level.
- The safety devices in this machine are:
 - › cap (safety valve)
 - › Pressure switch
 - › Thermostat on heating element.

Without water the thermostat interrupts power to the heater.

The machine is fitted with fuses for safeguarding use.

- **Warning!** During operation the cover and the gun steam outlet, can reach temperatures above 80°C. Don't put anything on the top of the machine.

5. TARGET

The steam generator has been designed and built for use by only dentists, dental technicians and goldsmiths. No other application are envisaged by Manufacturer who disclaims any responsibility for damage of whatever kind caused by improper use of the machine.

Modifications (however limited) , changing spare parts, modifying or use of non original spares, automatically release the manufacturer from any liability and shall cause the warranty to lapse.

Only for use by professionally qualified technician and trained adults.

6. PACKING

The machine is packed in a cardboard box and securely protected. On receipt of machine please check all is aligned with the order and is not damage. For any problems call the dealer or manufacturer immediately.

All our equipments undergoes final inspection and testing before leaving our factory.

Plastic funnel included.

Remove the packing material in accordance with prevailing environmental protection regulations.

7. TECHNICAL DATA

Boiler capacity	4,9 lt
Weight	14Kg
Pressure	4,5 bar
Absorbed Power	1500 W
Dimension	33,5 x 22 x 29.5 cm
Supply power	220V-50-60Hz

8. INSTALLATION

Before connection, check to see that the identification plate indicates the correct voltage for the electrical supply system. Installation must be carried out in compliance with all applicable safety standards and in accordance with the manufacturer's instructions.

The unit is safe when connected correctly to ground via an appropriate device in accordance with safety CEE standards. This fundamental requirement must be satisfied. For any doubt request an inspection by a professional qualified technician.

The manufacturer will not be held responsible for

- Damage caused by an improperly grounded system;
- Damage to objects and-or harm to people caused by improper installation.

9. USE

FILLING WATER

- Loosen the cap (safety valve).
- Fill the water by plastic funnel and do not exceed maximum water level.
- Tighten the cap.
- Place the rod supporting the steam gun in its seatings.
- Set the switch to on.
- Wait for the start light to go on (about 10-15 minutes).
- Push the button to activate the steam supply.

During use

- Check the water level on the visual indicator.
- Never open the safety cap when steamer is switched on.
- Condensation inside the pipe generates a small quantity of water which will exit at the first steam request.

- Check the water quantity by the glass level and if necessary fill it (fill it when the machine is switched OFF).
- Never use the machine with the water under the minimum level. If the water is under the level the light will become red. It is very important to pay attention to the level of the water.

If the minimum level of water is reached and you still require to use the machine, do so as follows:

- › Switch off and wait several minutes.
- › Loosen the cap little bit – ¼ turn and wait for the pressure to be released. Let the remaining pressure escape.
- › Remove the cap carefully.
- › Fill the boiler with hot water.

10. MAINTENANCE

ORDINARY MAINTENANCE and CLEANING

We suggest every month:

- Disconnect power supply.
- Loosen a cap and wait for the pressure to be released. Let the remaining pressure escape.
- Put 1/2 lt water with plastic funnel.
- Wait several minute, shake the boiler.
- Reverse down the boiler.
- If necessary try again it.

USE ONLY WATER

If broken or damage it is necessary contact a professional or dealer service and to provide and supply the spares with original parts.

The manufacturer is not responsible for any damage caused by inobservance of manual instruction, or damage caused by unauthorised modification, tampering, or the use of non-original or incompatible spare parts.

11. SPARES

For servicing please contact your local dealer or BA International with the following information:
Model, year of manufacture, serial number.

12. CE DECLARATION OF CONFORMITY

The company:

B.A. INTERNATIONAL LTD.
Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.

Declares on its own responsibility that tthe product conforms to the requisites established by the Directives below

MODEL	DIRECTIVE	YEAR
S5		2018
	2014/68/UE	2014/35/UE
	2014/30/UE	

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