B.A.INTERNATIONAL

# SuPreme S50

## Lab Unit

MODEL: BA190050  
BA190055  
BA190060  
BA190065  
BA190070

**Operating instructions**

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**Mode d'emploi**

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(manufacturer)

**B.A. INTERNATIONAL LTD.**

Unit 9, Kingsthorpe Business Centre,

Studland Road, Kingsthorpe,

Northampton NN2 6NE U.K.

Rev.2016/01

# Operating instructions

## LAB UNIT

### 1. Preface

We are pleased you have chosen a high quality product. These instructions enable you to become familiar with the unit and its features.

The technical specification, illustrations and dimension contained in this manual are not binding. No claims may be derived from them. We reserve the right to make technical improvements without amending these instructions.

### 2. Purpose

This product is appropriate to:

- Dental Technology
- Aviation Industry
- Industry
- Handicraft

**Any other kind of use is not allowed.**

Intended use includes the observance of all instructions of this manual as well as the compliance of care and maintenance.

**This product must be used indoors only.**

### 3. Service

For service please contact your local dealer or BA International:

[info@bainternational.com](mailto:info@bainternational.com)

For service inquiries please always indicate the serial number and the model.

### 4. Safety Instructions

4.1.

These instructions are to be carefully read and followed to ensure proper operation and handling of the unit. Failure to do so could result in serious injuries and/or damage to the unit. Any reproduction, copying, or transmission of these instructions is prohibited without the specific approval of the manufacturer.

4.2

The manufacturer cannot be held responsible for any damages resulting from:

- external interference (bad quality of the media or poor installation)
- improper use of information
- improper operation of unit
- improper repairs

Interference by an unauthorised party would nullify any implied warranty.  
The manufacturer recommends to use original spare parts only.

#### 4.3

A safe and proper operation of the unit can only be guaranteed by correct use of the relevant burs, as outlined in these instructions. When using rotary tools the limit given by the manufacturer must be observed. By using the reverse function the chuck can get loose while under heavy load!

#### **Accident prevention regulations must be observed!**

#### 4.4

During use, national legal regulations must be observed, in particular:

- The applicable health and safety regulations
- The accident prevention measures (risk of injury from rotating tools, eye injuries)
- Wear safety goggles
- Use protective screen
- Hazard due to incorrectly stored handpiece
- Injury caused by tightened rotary instrument
- Damage to the clamping system from dropping the handpiece

#### 4.5

Abnormal condition of the device or improper use may result in injury and threat of material and equipment.

#### **Unplug the unit before opening the housing.**

- The repair and maintenance of the electrical parts of the unit may only be done by professionally skilled and trained workers, informed of the safety guidelines. New spare parts, used in repairing the unit must be in accordance with the manufacturer's required specifications.
- The user has to check the safety function and the proper condition of the device before each use.

#### 4.6

Only use drills, milling and grinding tools acc. to DIN EN ISO 1797-1!

Never use drills, milling- or grinding tools with worn shaft!

Respect the use-speed instructions of the manufacturer!

## **5. Electronics and Electrical Equipment Disposal**

Based on the EC Directive 2002/96 on Waste Electrical- and Electronic Equipment we point out that this product is subject to that Directive and must be performed to a particular waste disposal within Europe.

WEEE-Reg.-Nr. DE 81870190

## **6. Transportation Damages**

1. The recipient notes the loss or damages immediately.  
The recipient informs the delivery agent immediately about the damage.
2. Keep the product and the packaging unchanged.
3. Do not use the product.
4. Report the loss to the delivery agency.
5. Report the damage to the manufacturer
6. Return the damaged product only after consultation the manufacturer.

## 7. Device Overview BA

S50F BA Supreme Lab Unit (Foot Control)

S50T BA Supreme Lab Unit (Table Control)

S50K BA Supreme Lab Unit (Knee Control)

S50N BA Supreme Lab handpiece

S50H BA Supreme heavy duty Lab handpiece


## 8. Operation Control Units

### All types of control units

Check if the network data correspond to the data on the label. Plug the power cord into the power jack 1 on the device and then put the power plug into the socket. Insert the handpiece cable into the connector 5 and screw together.


### S50K BA Supreme Lab Unit (Knee Control)

The unit needs to be turned "on" with the on/off switch ,2'. The red light turns on. Select the desired max. speed with the speed-selector knob ,3'. The rate of revolutions will be continuously variable by activating the knee lever ,4' into the direction of the max. speed. By using the reversal rotation switch ,7' (press approx. 1 sec.), the direction of course of the motor hand piece changes. The max. speed of the counter clockwise rotation is always limited to 30.000 rpm. The speed change from 30.000 rpm to 50.000 rpm is done by activating the speed selection knob ,6' (please press 1 sec).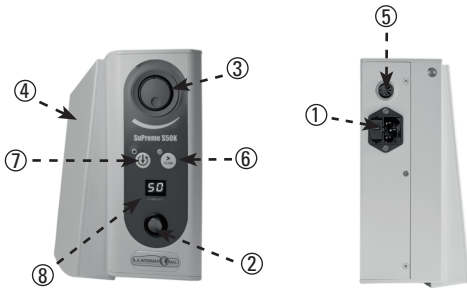


### S50T BA Supreme Lab Unit (Table Control)

The unit needs to be turned „on“ with the on/off switch ,2'. The red light turns on. Select the desired max. speed with the speed-selector knob ,3'. By activating the treadle switch ,10', which is plugged into the back of the unit, the selected rate of revolution will be reached. In order to operate the unit without the treadle switch, you need to unplug it out of the connector ,11'. The unit operates at the set max. speed. By using the reversal rotation switch, the direction of course of the motorhandpiece changes. The max. speed of the counter clockwise rotation is always limited to 30.000 rpm. The speed change from 30.000 rpm to 50.000 rpm is done by activating the speed selection knob ,6' (please press 1 sec).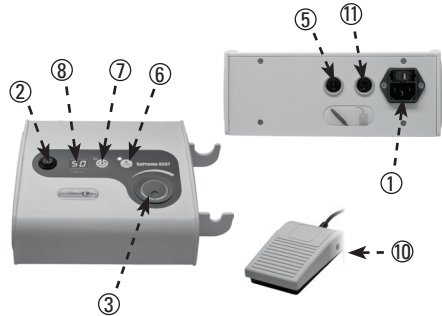

### S50F BA Supreme Lab Unit (Foot Control)

The unit needs to be turned "on" with the on/off switch ,2'. The red control lamp lits. The rate of revolutions will be continuously variable by activating the foot lever ,9' into the direction of the max. speed. The motorhandpiece stops by releasing the foot lever. The speed change from 30.000 rpm to 50.000 rpm is done by activating the speed selection knob ,6' (please press 1 sec).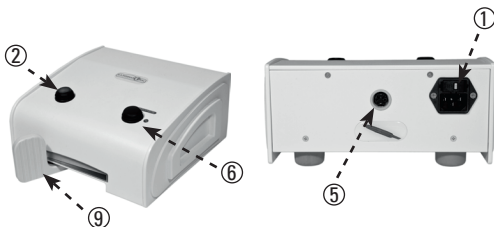

### Overload protection of the motors

In case of an overload the built-in electronics will shut itself and the motor itself off. By activating the knee-lever, the foot pedal or the treadle switch the motor hand piece is immediately ready for use.


## 9. Operation Handpieces

The relevant motor hand piece needs to be plugged in and locked at the back side of the control unit. To open the mechanism (for the chuck) you need to turn the grip sleeve clockwise in order to exchange the desired rotating instrument. In order to close it again, you need to turn the grip sleeve counter clockwise until it snaps.

### Attention:

The tool change may only be performed with the engine off.

When the motorhandpiece is not operating a tool or the supplied pin has to be clamped in.

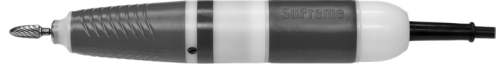S50N BA Supreme Lab handpiece


S50H BA Supreme heavy duty Lab handpiece

## 10. Handpiece Care and cleaning

### Cleaning and changing the chuck

#### 1. Remove screw:

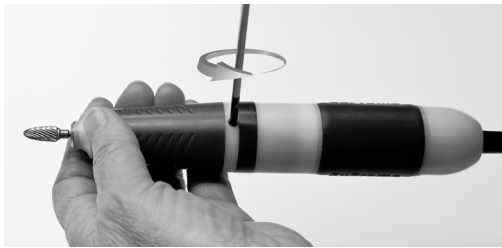- Remove slotted screw with a screwdriver

#### 2. Open handpiece:

- Open handpiece in direction of arrow
- Remove the tool

#### 3. Pull off the dust hood:

- Pull forward the dust hood

#### 4. Untighten the chuck:


- Lock the shaft with a bit
- Unscrew the chuck with chuck key 100.0017 as shown

5. Remove the chuck

- *Remove the chuck*

6. Clean the chuck: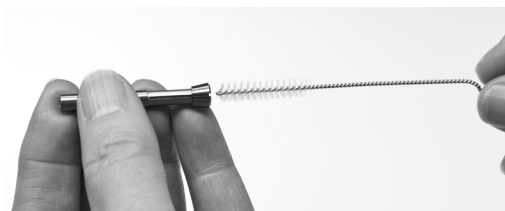

- *Clean the chuck with a brush*

7. Insert the chuck:

- *Insert the cleaned chuck*

8. Tighten the chuck:

- *Lock the shaft with a bit*
- *Fasten the chuck with the chuck key 100.0017 as shown*

9. Fix the dust hood:


- *Pull on the dust hood*

10. Close handpiece

- *Insert the bit*
- *Close the handpiece in direction of arrow*

11. Fix the screw: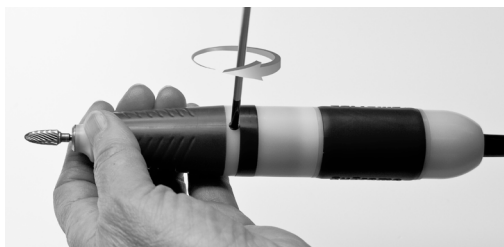

- *Rescrew the slotted screw with a screwdriver*

## 11. Cleaning + Care Handpieces HD

### Cleaning and changing the chuck

1. Remove the tool:

*Open the chuck by turning the handpiece grip sleeve and simultaneously retaining the motor grip sleeve until it clicks. Remove the tool.*

**Never open the handpiece during operation.**

2. Remove motor end cap:

*Screw off the motor end cap. Hold the motor grip sleeve. Screw off the motor end cap and pull off the plug carefully.*

3. Untighten the chuck:

*Hold the shaft with the socket and screw off the chuck with the chuck key.*

4. Remove the chuck:


*Remove the chuck out of the shaft, check, clean or change the chuck. Defective or polluted chucks reduce the tension and the concentricity.*

5. Clean the shaft:

*Clean the shaft with a brush.*

6. Clean the chuck: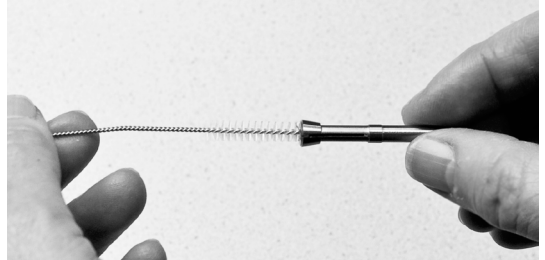

*Clean the chuck with a brush.*

**7. Insert the chuck:**

*Insert the chuck.*

**8. Tighten the chuck:**

*Hold the shaft at the motor with the socket and screw in the chuck with the chuck key.*

**9. Assemble motor end cap:**

*Insert the plug onto the contacts. Screw on the motor end cap and tighten firmly by hand.*

**10. Tight the tool**

*Set in the tool and close the chuck by turning the handpiece grip sleeve and hold simultaneously the motor grip sleeve. Pay attention to the click. The handpiece is ready for operation.*

Itemnumber### 100.0017

Supreme Handpiece  
S50N  
Ø 2,35

Supreme Handpiece  
S50N  
Ø 3

Supreme Handpiece  
S50H  
Ø 2,35

Supreme Handpiece  
S50H  
Ø 3

## 12. Troubleshooting Guide

### Control Unit

Error E2	a.) The spindle has no electrical connection to the control unit b.) Spindle is blocked c.) Spindle does not turn easily	a.) Maybe broken cable or motor pins damaged b.) Chuck and spindle are opened c.) Bearings are defective
Error E3	a.) Motor defective b.) Motor draws too much current c.) Handpiece is blocked d.) Function no longer available	a.) Typical overload error b.) Motor defective, maybe dropped c.) Chuck is not closed d.) Chuck and bearings are very dirty
Typical user error: handpiece is too heavily loaded in a specific speed range (wrong tools for wrong speed).		

### Control Unit

On-off switch does not lit	Check the fuse and eventually change 230V/2AT, 115V/4ATT
Foot pedal has not function	Check the control unit without foot pedal. Eventually return the foot pedal to After-sales-service

### Handpiece

Handpiece heated strongly	Bearings are eventually defective (return to aftersales-service)
Handpiece not correctly closed	Close the handpiece
Chuck is loose	Tighten the chuck
Handpiece has running noise	Bearings are eventually defective (return to aftersales-service)
Chuck beats or does not hold the tools correctly	Clean the chuck Change the chuck

## 13. Maintenance and Care

The chuck has to be cleaned regularly (approximately month or for heavy dirt works approximately weekly)

### Attention:

**Do not treat the handpieces with compressed air or oil.**

## 14. Technical Data

Dimension: Control unit	Supreme S50K	Supreme S50T	Supreme S50F
Width	300mm	220mm	220mm
Height	100mm	100mm	100mm
Depth	230mm	200mm	250mm
Weight	3,3kg	3,5kg	3,5kg
Speed min.	1000min <sup>-1</sup>	1000min <sup>-1</sup>	1000min <sup>-1</sup>
Speed max.	50.000min <sup>-1</sup>	50.000min <sup>-1</sup>	50.000min <sup>-1</sup>
Voltage	AC 220-240V or 100-100V	AC 220-240V or 100-100V	AC 220-240V or 100-100V
Input power	280W	280W	280W
Rated frequency	50/60Hz	50/60Hz	50/60Hz
Protection class	I	I	I
Dimension: Handpiece	Supreme S50N	Supreme S50H	
Length	156mm	159mm	
Diameter max.	28mm	30mm	
Torque	7,8Ncm	8,9Ncm	
Weight	220g	389g	

## 15. Terms of Warranty

The above mentioned product is of the highest quality and workmanship. This unit carries following warranty under our warranty conditions. Decisive for the warranty time will be the date of sales invoice.

### 24 months warranty

Defects (material, construction or productions defects) which occur within the time of warranty will be repaired free of charge. Replaced parts will be retained by the manufacturer. The units must being returned to the manufacturer without occurring costs. We decline any other kind of liabilities. The initial warranty time will not be extended by replace parts/unit. The following defects are not covered by the warranty:

- Improper operation or handling
- Parts, which succumb to natural wear
- Outer damages due to damage in transit, and damage by impact/knock
- Repairs and alterations which were done by unauthorised persons
- Non-observance of operating, maintenance or connection rules
- Damage from corrosion, contamination or chemical or electrical influences which are abnormal or not allowed according to factory specifications
- The above mentioned warranty period refers to a daily working time of 8 hours

The warranty generally does not cover rubber parts and the colour fastness of plastic parts.

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