B.A.INTERNATIONAL

BA Ultimate Straight Handpiece

Ultimate BA170 REF. 9792870 External Spray System
Ultimate BA170T REF. 9792924 External Spray System
Ultimate BA280 REF. 9792871 Internal Spray System

Operation Manual

AUTOCLAVABLE
Thermo-Disinfectable
ENGLISH
OM-HK114E 003

CE The EU directive 93/42/EEC was applied in the design and production of this medical device.

Caution before use


- The BA Brand Ultimate straight handpiece is intended for use by dental professionals only.
- Read this Operation Manual carefully before using the straight handpiece. Be sure to follow all the operating instructions, including all cautions and warnings.
- Keep this Operation Manual for future reference.

Specifications

	BA170	BA170T	BA280
Speed Ratio	1:1 Drive		
Allowable Maximum	40,000 rpm		
Speed	2.35 HP bur		
Chuck Configuration	Doriot type		Non-doriot type
Nose Configuration	Autoclavable		Autoclavable
Sterilization	Thermo-Disinfectable		Thermo-Disinfectable


Note : The maximum speed may depend directly on the type of tubing and type of Motor.

Parts description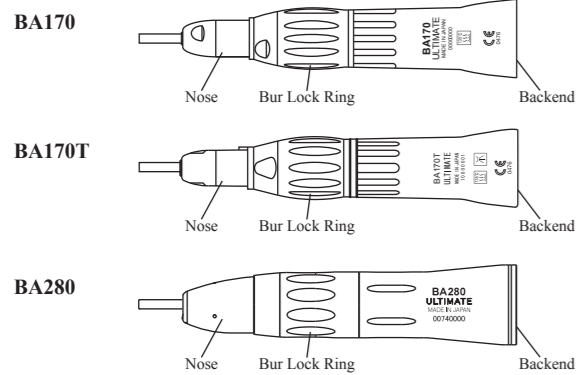

Removal/Insertion of Bur from/into Straight Handpiece

- Holding straight handpiece, twist bur lock ring in the direction indicated by the black arrow until the chuck clicks open and remove the existing bur blank.
- Insert bur spindle into chuck all the way until it touches the back of the chuck.
- Place the straight handpiece upward so that a bur inserted does not slide down, and then twist the bur lock ring in the direction indicated by the white arrow until the chuck clicks shut.

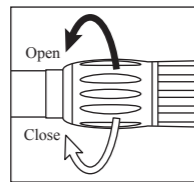Warning

- Make sure to align the ● mark with "●" mark (BA170/BA170T) or "C" (BA280). If those marks are not completely aligned, it may cause handpiece overheating or patient burn injuries.
- Never use a bur with the bent shank or a damaged bur for the straight handpiece. Such a bur may not only injure the patient but critically damage the straight handpiece.

Caution

- Before use, make sure that a bur is securely locked, pulling it gently with fingers. Make a trial run.
- Never use a non-ISO standard bur for the straight handpiece.
- Never open/close the chuck without inserting a bur into the chuck hole, as the chuck holding intensity may be prematurely reduced.

Installation of Straight Handpiece onto Motor

- Make sure that the motor onto which the straight handpiece is attached is an ISO E-type.
- Insert the E-type fitting of the motor into the back-end of the straight handpiece.
- Make sure that the straight handpiece is securely installed onto the motor, by pulling it gently with fingers, and then make a trial run before use.

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Warning


- If the straight handpiece cannot be smoothly attached/removed onto/from the motor or if the straight handpiece rattles after installation, never operate the straight handpiece.
- Whenever abnormality in performance, sound level, heat level, function, etc. is sensed, stop using the straight handpiece immediately and consult **BA International Ltd. or an accredited BA certified Service Centre.**
- Never install/remove the straight handpiece onto/from the motor while the motor is in operation.

Caution

- Never operate the straight handpiece by installing it onto a motor which runs beyond the allowable maximum speed, as the excess speed may cause critical damage to the internal mechanism.
- Never dismantle or alter the straight handpiece.

Lubrication

Use **BA ULTIMATE SPRAY**. The straight handpiece must be lubricated once a day or before autoclaving using the following procedures:

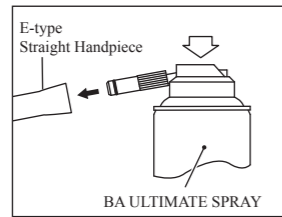- Remove straight handpiece from motor.
- Remove the bur.
- Wipe off the saliva, dirt, etc. stuck to the surface of the straight handpiece with an alcohol-soaked soft tissue or cotton swab.
- Keep the straight handpiece in a horizontal position and inject BA ULTIMATE SPRAY for three seconds through the back-end of the straight handpiece using the appropriate (E-Type) nozzle.
- Install the straight handpiece onto the motor after the spray injection.
- Make a trial run for five seconds.
- Wipe down the excess lubricant on the surface of the straight handpiece with a soft tissue.

Caution

- Never use a wire brush or disinfectants for surface cleaning.
- Never immerse the straight handpiece in an ultrasonic cleaner or disinfectant solution.
- Lack of lubrication after use or before autoclaving may cause premature failure of the straight handpiece.

Sterilization

BA handpiece with this symbol can be sterilized at 134 degrees in an autoclave.

- Before sterilization, clean the surface of the handpiece.
- Lubricate the handpiece, and then make a trial run.
- Remove the handpiece from the motor.
- Put the handpiece into a sterilization pouch when using a vacuum type autoclave. Do not combine it with other instruments in the same pouch.
- Autoclave the handpiece at 121 degrees C for 20 minutes or 134 degrees C for 15 minutes.

BA handpiece with this symbol can be cleaned and disinfected in a thermo washer disinfectant.

- Follow manufacturer's operating instructions including caution, warning, etc. in use.

Warning

- Never use dry heat sterilization, cold sterilization, etc.
- Never autoclave the straight handpiece together with others in a sterilization pouch.

Service and Repair

Important

This straight handpiece is a result of precision engineering and cannot be dismantled. For service and repair, always send it to either **BA International Ltd.** or an accredited **BA certified service centre** without dismantling it.

B.A. INTERNATIONAL LTD.
Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.
Rev.2016/08

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