BA-Optima

Dental High- Speed Handpiece Operation Manual

1. Product Description

BA535FB/FM

BA535

- Connection (Borden 2-hole, Midwest 4-hole)
- Head size 535: torque
- Triple Spray hole
- Push button

2. Specifications

Fixed Connection

	BA535FB	BA535FM
Connection	Borden 2-hole	Midwest 4-hole
Spray Hole	Triple	Triple
Rotation speed (rpm)	T≥ 300,000 M≥ 350,000	
Bur size-diameter/length (mm)	Ø 1.59~Ø 1.60 / T(19-25); M (16-21)	
Drive air pressure (bar)*	2.2~2.8	
Suggested drive air pressure (bar)*	2.5	
Water flow rate (ml/min)	>50	
Water pressure (bar)	0.8~2.0	
Air pressure (bar)	1.0~3.0	

- The ISO 2/4 holes is defined in accordance with ISO 9168:2009 Measured by multi gauge
- Air pressure must be higher than water pressure. BA535: torque head

Air compressor and other devices used by the handpiece must be in accordance with ISO requirement (MUST set air pressure higher than water pressure before operation.).

1. The use of contaminated water / compressed air containing moisture, impurity or oil to drive air could shorten the lifetime of the cartridge/rotor, and block the spray nozzle.
2. If the required configuration is not followed, they will be a risk of damaging the devices and/or cause potential injury to patient.
3. After unpacking the handpiece from the packaging or before operating, please follow the sterilization process from step 7 to 9 : Cleaning>lubrication>sterilization.

3. Safety Notes

1. If the handpiece is damaged (ie deformed caused by dropping) or improper use (ie sterilized above suggested temperature), immediately stop using.
2. Before operation, always make sure the product and its accessories meet the specified requirements (ie drive air pressure. lubrication...etc).
3. Please contact BA International or our approved dealer with proof of purchase, whenever you have problems such as abnormal vibration, exceeding temperature or unusual noise...etc.
4. Avoid contact between the instrument head and soft tissue.

4. Installation

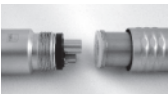
Fig.1

Fig.2

Fig.3

To install handpiece into ISO 2/4-hole hose(*), please follow the steps illustrated from Fig.1 to Fig.3. Make sure the handpiece is connected to the hose tightly. If there is any leakage of water or air, please repeat this installation procedure.

* The holes hose is defined in accordance with ISO 9168 : 2009

5. Changing Bur

1. Follow arrow direction shown in Fig. to remove or take off or install the bur.
2. Put on protective gloves to reduce the risk of infection, do not touch the used burs without proper protection.
3. Make sure bur is tightly secured before operation, otherwise risks of injury to doctors, patients, or the third party could occur.
4. Bur must comply with ISO requirement as described in the Spec.
Do not use burs which are not in good condition such as bent or damaged. Use of such burs can decrease the lifetime of the handpiece and can cause severe consequence such as injury to patient.
5. Do not press the button while bur is rotating, otherwise it could harm internal parts or cause an injury.

6. Service of your handpieces

Please contact BA International or your local BA dealers for the service and repair of your products. Please note this is a medical device and can only be repaired or maintained by a qualified and fully trained technician.

7. Cleaning

1. Put on protective gloves to detach the handpiece from the hose or coupling.
2. Use clean cloth with mild cleaning solution to clean the handpiece surface carefully.
3. Do not immerse handpiece or cartridges in disinfectant solution or water; Do not clean by ultrasonic process.
4. Cleaning solution must be non-erosive and harmless to human. Solution with 80% ethyl alcohol is recommended. Please also follow recommendation of the cleaning solution manufacturer for suggested usage.
5. Clean the spray nozzle with the nozzle cleaner needle when it is blocked to prevent burning caused by overheat of the tooth caused by overheating (see above figure).

8. Lubrication and Maintenance

1. Use BA Ultimate Spray for maintenance, any unspecified oil not intended for lubricating handpieces could shorten the lifetime of the handpiece and harm human body.
2. To lubricate BA535 series handpiece, put Spindle Nozzle into nozzle of the lubrication oil can and inject the oil into drive air tube for 2~3 seconds (Fig.4).
3. Be sure to lubricate the handpieces prior to sterilization and run the handpieces for 20~30 seconds to remove residual oil and dirt.
4. To lubricate handpiece chucks, put spindle nozzle into nozzle of the lubrication oil can and inject lubrication oil into the chuck (Fig.5) for 1 second. It's recommended to lubricate chuck at least once a week and then proceed to step 2.
5. After unpacking and prior to first use, be sure to lubricate with oil and run the handpieces at low speed for 30 seconds and in full speed for 20~30 seconds to emit residual oil.

Fig.4

Fig.5

9. Sterilization

BA International suggest sterilization in accordance with EN13060

1. Place handpiece in single-use sterile bag and proceed to steam prevacuum sterilization. Do not open the sterile bag before use.
2. Use qualified sterilization equipment specified by the authority/government for sterilization procedures.
3. Do not use chemical steam nor immerse handpiece or cartridge/rotor in disinfectant solution, otherwise could damage the handpiece and cause severe consequences.
4. Sterilization holding time a minimum of 4 minutes at 134°C. (drying time is 15 minutes)

10. Disclaimer

1. Information in this operation manual was carefully reviewed. We cannot assume any liability for any damage arising from improper use of the handpiece or from use of this manual.
2. Products are subject to change without notice.
3. This product is only for dental treatment and use for any other purpose may damage the handpiece or shorten the lifetime of the product.

11. Quality and Service

Our products have passed strict Q.C. procedure. Please note improper use, not using the recommended parts and accessories or modifying our products without our approval may shorten the lifetime of the product or cause damage. Please contact BA International or our approved dealers with proof of purchase for service and any after sales queries.

B.A. INTERNATIONAL LTD.
Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.
Rev.2016/08

BA-Optima

Dental High- Speed Handpiece Operation Manual

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