BA OPTIMA


Operation Manual for:

- BA520FM
BA600068
- BA520FB
BA600088


CE 0197 Made in Japan

OM-TK519E 000


Please read this operation manual carefully before use, and file for future reference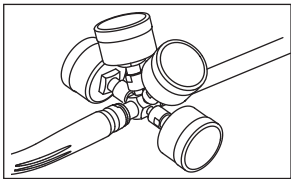


1

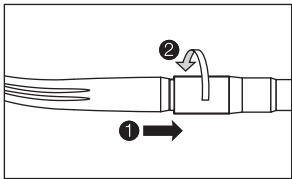
2

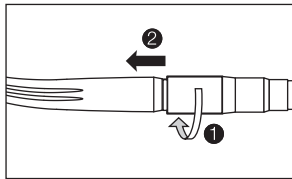
3

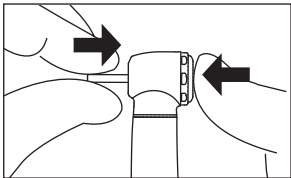
4

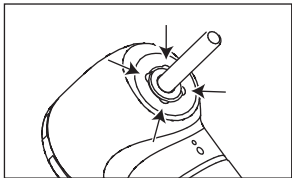
5

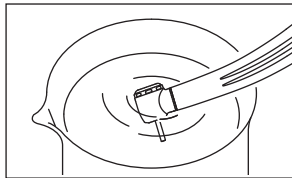
6

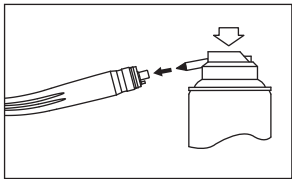7

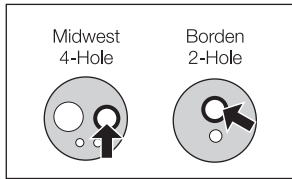8

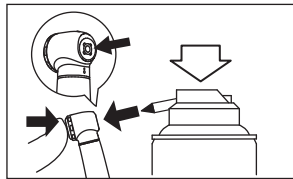9

1. User and Intended Use

User : Qualified Professionals (Dentist)

Intended Use : This device is an air-powered dental handpiece intended for use in general dental applications such as removing carious material, reducing hard tooth structure, cavity and crown preparation, removal of fillings, finishing tooth preparations and restorations and polishing teeth.

2. Precautions for handling and operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
 WARNING	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.
 CAUTION	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.
NOTICE	General product specification information highlighted to avoid product malfunction and performance reduction.

WARNING

- **Depressing the Push Button while the handpiece is in rotation may lead to overheating, causing burn injuries or product failure. Avoid the push button to contact with any oral tissue.**

CAUTION

- Read this Operation Manual before use to fully understand the product functions and file for future reference.
- When operating the product always consider the safety of the patient.
- Users are responsible for the operational control, maintenance and continual inspection of this product.
- Do not attempt to disassemble the product nor tamper with the mechanism except as recommend by B.A. International in this Operation Manual.
- Do not allow any impact on to the product. Do not drop the product.
- Operators and all others in the area must wear eye protection and a mask when operating this handpiece.
- Should the product function abnormally, cease operation immediately and contact your Authorized B.A. International Dealer.
- Do not use high acid water or sterilizing solutions to wipe, immerse or clean the product.
- The products are delivered in a non-sterile condition and must be

autoclaved prior to use.

- Perform regular function and maintenance checks.
- If the product is not used for a long period check it is functioning correctly before using on a patient.
- To avoid clinical downtime it is recommended that a spare be kept on hand in case of a breakdown during surgery.

3. Setting of Air & Water Supply Pressure

Measure the supply pressure at the handpiece / hose connection point and set the pressure to the value specified on the specification table.

(Fig.1)

For Multi Gauge information refer to Option Parts List.

WARNING

- **Do not exceed the optimum pressure specified on the specification table.**

CAUTION

- Do not use air contaminated by dust, moisture and oil.

4. Connection & Disconnection of the Handpiece

Refer to Operation Manuals of coupling and hose before connecting the handpiece.

4-1 Connection

- 1) Insert the handpiece correctly into the hose connector and tighten the hose nut. (Fig.2)
- 2) Make sure the handpiece is connected firmly to the hose.

4-2 Disconnection

Loosen the Hose Nut and remove from the Hose. (Fig.3)

5. Insertion & Removal of the Bur

5-1 To Insert the Bur

- 1) Insert the bur until it is correctly seated in place. (Fig.4)
- 2) Depress the Push Button and insert the bur into the chuck until it is secure then release the button.
- 3) Ensure that the bur is secure by gently pulling and pushing the bur WITHOUT depressing the Push Button.

5-2 To Remove the Bur

Depress the Push Button firmly and remove the bur.

NOTICE

- Grip the handpiece while placing your thumb tip on the Push button which makes it easier to depress the button.

CAUTION

- Always insert the bur all the way into the chuck.
- Remove the bur only after the handpiece has completely stopped rotating.
- Always keep the bur shank clean. Entry of debris into the chuck, via the bur shank, could cause bur rotation slip and also prevent the bur from being securely located in the chuck.
- Do not exceed the bur speed recommended by the bur manufacturer.
- Do not exceed maximum bur length recommended by the handpiece manufacturer.
- Do not apply excess pressure to the bur as it may break or bend or become difficult to remove.
- DO NOT use burs with problems listed below as the bur may break or disengage from the chuck.
 - Bent, deformed, anisomorous (worn), rusted, broken, deficient bur.
 - Bur which is cracked on the edge or axis.
 - Non-ISO standard, or tampered bur.

6. Check before treatment

Check that the Head Cap is firmly tightened. Also check for handpiece vibration, noise and overheating. If any abnormalities are found do not use the handpiece and contact your Authorized B.A. International Dealer.

7. Maintenance

After each patient maintain the product as follows.

7-1 Cleaning handpieces with the Clean-Head System

After the treatment of each patient, clean the Clean Head.

- 1) Remove dirt and debris from the Clean Head Holes with the brush. (Fig.5)
- 2) Half fill a cup with clean water.
- 3) Rotate the handpiece and immerse half of the handpiece head in the cup of water. (Fig.6)
- 4) Rotate then stop intermittently the handpiece 3 times for 2 to 3 seconds each time.

5) Wipe the handpiece dry.

*If the dirt could not be removed from the hole, clean it by brush.

7-2 Cleaning (Handpiece)

- 1) Remove dirt and debris from the product. Do not use a wire brush.
- 2) Wipe clean with alcohol-immersed cotton swab or cloth.

 This icon denotes that the product can be washed via Thermo-Disinfector.
Refer to the Thermo-Disinfector manual.

CAUTION

- After washing with Thermo-Disinfector and prior to lubrication, dry the product until all internal moisture is thoroughly removed. Thermo-Disinfector moisture remaining inside the product could reduce the effect of lubrication and could cause corrosion inside of the product.
- To clean the product never use any solvent such as benzine or thinner.

7-3 Lubrication

■ BA ULTIMATE SPRAY

Apply BA ULTIMATE SPRAY every time after each use and/or before autoclaving.

- 1) Remove the handpiece from the hose.
- 2) Remove the bur from the handpiece.
- 3) Mount the Tip Nozzle into the spray can port. (Fig.7)
- 4) Insert the Tip Nozzle into the Drive Air Port of the handpiece. Hold the handpiece and spray for approximately 2-3 seconds. Apply lubricant until it expels from the handpiece head for at least 2 seconds. (Fig.8)

CAUTION

- When applying spray be sure to hold the handpiece firmly to prevent the handpiece from slipping out of the hand due to the spray pressure.
- Hold the spray can upright.

■ Chuck cleaning

Clean Push Button chuck once a week.

- 1) Mount the Tip Nozzle into the spray can port.
- 2) Lubricate the chuck directly through the bur insertion hole. (Fig.9)
- 3) Lubricate the handpiece by using BA ULTIMATE SPRAY. (Fig.7)

CAUTION

- If the chuck is not regularly cleaned the chuck grip may be weakened and the bur may be accidentally released while in operation.

7-4 Sterilization

Sterilize the product by autoclave sterilization. Remove the bur after each patient and sterilize as noted below.

- 1) Insert into an autoclave pouch. Seal the pouch.
- 2) Autoclavable under the conditions below.
Autoclave for more than 15 min. at 132°C, or 3 min. at 134°C.
- 3) The handpiece should remain in the autoclave pouch until required for use.

CAUTION

- Do not autoclave the product with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments.
- Keep the product in suitable atmospheric pressure, temperature, humidity, ventilation, and sunlight. The air should be free from dust, salt and sulphur.
- Immediately after use, the product should be cleaned, lubricated and sterilized. If blood remains on the external or internal surfaces it can become clotted and cause rust.
- Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product.
- If the sterilizer chamber temperature may exceed 135°C during the dry cycle then delete the dry cycle.
- Autoclave sterilization is recommended for the product. The validity of other sterilization methods is not confirmed.
- Do not touch the product immediately after autoclaving as it will be very hot and must remain in a sterile condition.

NOTICE

- B.A. International recommends Class B sterilizers as stated in EN13060.

8. Periodical Maintenance Checks

Perform periodical maintenance checks every three months, referring to the check sheet below. If any abnormalities are found, contact your Authorized B.A. International Dealer.

Points to check	Details
Rotation	Rotate the handpiece and check for abnormalities such as abnormal rotation, vibration, noise, and overheating.
Coolant Water	Operate the handpiece and check that the coolant water is flowing through all spray ports.

9. Symbol

This product is Autoclavable up to Max.135°C.

This product can be washed via Thermo Disinfector.

Conforms to CE European Directive of "Medical device directive 93/42/EEC."

Manufacturer.

Authorized representative in the European community.

Consult operation instructions.

10. Warranty

B.A. International products are warranted against manufacturing errors and defects in materials. B.A. International reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose or has been tampered with by unqualified personnel or has had non B.A. International parts installed. Replacement parts are available for seven years beyond discontinuation of the model.

11. Option Parts List

Model	Order Code	Compatible Product
BA Multigauge for Midwest connections	BA08010030	BA520FM
Borden to Midwest Adaptor	TST790	BA520FB + BA Multigauge

12. Disposing product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled industrial wastes, to dispose the product for you.

13. Specifications

Model	BA520FM	BA520FB
Hose Connection Type	ISO 9168 Type 2 (Midwest 4 hole)	ISO 9168 Type 1 (Borden 2 hole)
Rotation Speed	350,000 - 450,000min ⁻¹	
Bur Type	ISO 1797-1 ϕ 1.59 - 1.60mm Standard Bur	
Chucking Length	10.2mm	
Max. Bur Length	21mm	
Max. Working Part Diameter	ϕ 2mm	
Drive Air Pressure	0.20 - 0.25MPa (2.0 - 2.5kgf/cm ²)	
Air Consumption	\leq 55 NL/min (0.25MPa)	
Water Pressure	0.05 - 0.20MPa (0.5 - 2.0kgf/cm ²)	
Chip Air Pressure	0.15 - 0.25MPa (1.5 - 2.5kgf/cm ²)	-
Use Environment	Temperature: 0 - 40°C (No Condensation), Humidity: 30 - 75%	
Transportation and Store Environment	Temperature: -10 - 50°C, Humidity: 10 - 85%, Atmospheric Pressure: 500 - 1,060hPa	

14. Service and Repair

Important

This high-speed turbine is a result of precision engineering and cannot be dismantled. For service and repair, always send it to either BA International Ltd. or an accredited BA certified service centre without dismantling it.

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NSK Europe GmbH
B.A. INTERNATIONAL LTD.