BA ULTIMATE


BA 695
BA 695FBA
BA 888FM

English / Deutsch / Français / Español / Italiano / Polski







Operating Instructions
Gebrauchsanweisung
Notice d'utilisation

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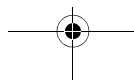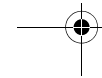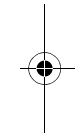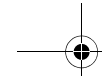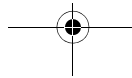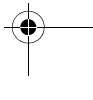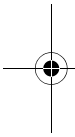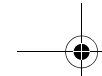
6556083_Titel.fm Seite 2 Dienstag, 22. November 2016 11:50 11



BA ULTIMATE

Dental Air Turbine Handpiece with fixed connection



BA 695

BA 695FBA

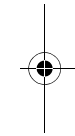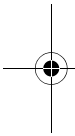BA 888FM

Operating Instructions

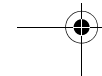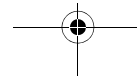English



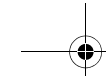
Table of contents

Operating Instructions BA ULTIMATE



Table of contents

1	Before you begin	5
1.1	Structure of the document	6
1.2	Service life	9
2	Safety information	10
3	Technical description	12
3.1	Task	12
3.2	Design of the BA ULTIMATE with fixed connection	12
3.3	Technical data	13
3.4	Turbine connection types available	16
3.5	Production data	16
4	Preparation	17

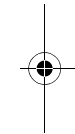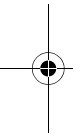2

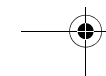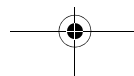65 56 083 D3560.201.10.03.02

Operating Instructions BA ULTIMATE

Table of contents

4.1	Initial start-up and longer breaks in use	17
4.2	Prior to starting the work day.....	17
4.3	Before each patient	18
5	Operation	19
5.1	Adjusting the cooling spray	20
6	Follow-up	21
6.1	After each treatment session.....	21
6.2	At the end of the work day.....	22
7	Conditioning	23
7.1	Conduct pre-disinfection.....	23
7.2	Automated cleaning and disinfecting.....	24
7.3	Manual cleaning and disinfection	26

English





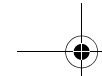
Table of contents


Operating Instructions BA ULTIMATE


8

7.4	Manual maintenance	28
7.5	Sterilizing	32
	Maintenance	34
8.1	Testing the FG clamping system	34

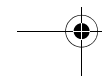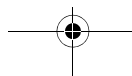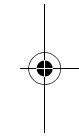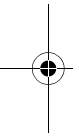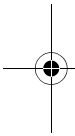Intended Use

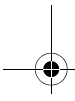65 56 083 D3560.201.10.03.02

English

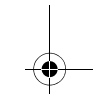
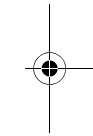
1 Before you begin ...

The BA 695 / BA 695FBA / BA 888FM turbine corresponds with the provisions of state-of-the-art technology. The BA 695 / BA 695FBA / BA 888FM turbine complies with the ISO 14457 and ISO 9168 standards.

1. Read the operating instructions before using the BA 695 / BA 695FBA / BA 888FM turbine.
2. Use the BA 695 / BA 695FBA / BA 888FM turbine only for the purposes described in the operating instructions.
3. Follow the hygiene regulations, work safety regulations, and accident prevention measures that apply to the BA 695 / BA 695FBA / BA 888FM turbine.

The BA 695 / BA 695FBA / BA 888FM turbine is used to treat dental disease and injuries.

The treatment involves the rotary processing of hard and soft tooth substance and dental prostheses (crowns, fillings, bridges, etc.).

1 Before you begin ...

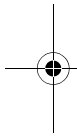
Contraindications

Target group

None

This product is intended only for use by trained dental personnel in dental practices and laboratories.

Operating Instructions BA ULTIMATE

Warnings

1.1 Structure of the document

1.1.1 Labeling of information

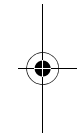
➤ To prevent injuries, please observe all warnings.

Warnings are labeled as follows:

⚠ **DANGER!** indicates a danger **leading** to death or serious injury if not avoided.


⚠ **WARNING!** indicates a danger that **may lead** to death or serious injury if not avoided.

⚠ **CAUTION!** indicates a danger that **may lead** to injury if not avoided.

6



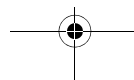
65 56 083 D3560.201.10.03.02


Operating Instructions BA ULTIMATE

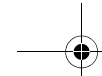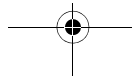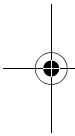Instructions for use


1 Before you begin ...

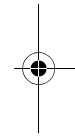English

7

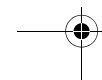➤ To prevent material damage and additional expenses, please observe all instructions for use.

Instructions for use are labeled as follows:

NOTICE! indicates measures for the prevention of material damage.

IMPORTANT: indicates information on the avoidance of additional expenses and other important information.






Tip: indicates information for facilitating work.






1.1.2 Formats and symbols

The formats and symbols used in this document have the following meaning:

<ul style="list-style-type: none"> ✓ Prerequisite 1. First action step 2. Second action step or > Alternative action ↵ Result ➤ Individual action step 	Requests you to do something.
Use of formats and symbols [→ 8].	Identifies a reference to another text passage and specifies its page number.
• List	Identifies a list.

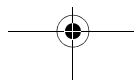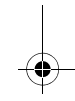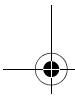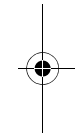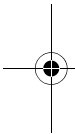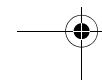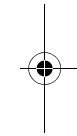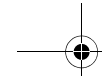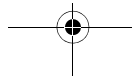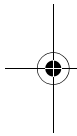
1.2 Service life

When used as intended:

- Non-moving parts have a typical service life of 5 years
- Moving parts have a typical service life of 3 years

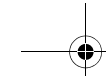
No warranty claim can be inferred here, as wear may occur earlier or later than indicated above depending on use, frequency of sterilization, and frequency of maintenance.

2 Safety information


Obligations of the user

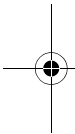- Use only fault-free materials that do **not** deviate from the specified data [→ 13].
- Protect yourself, patients, and others against any foreseeable dangers. To do this, follow the safety information.
- Comply with the Intended use of the equipment.
- You should always keep these operating instructions within reach for further reference.

Preventing the spread of infections and cross contamination

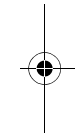Prevent the spread of infections and cross contamination between patients, users, and third parties. Sterilize equipment after each patient. Take the appropriate hygiene measures, e.g. wear protective gloves.



Malfunction or damage

Discontinue use immediately in case of malfunction, unusual or different sounds or damage. Damaged turbines may cause injury. Notify the dental dealer or the manufacturer.

Potential explosion hazard

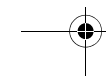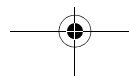
Do **not** use this product in areas subject to explosion hazards.


Operating Instructions BA ULTIMATE

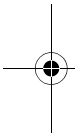Repair

Spare and accessory parts

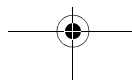2 Safety information

English

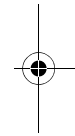
Do **not** attempt to repair the turbine yourself.

The BA instrument is the result of a precise development and must **not** be disassembled by third parties. In the case of service or repair work, please send the instrument in an assembled state to B.A. International Ltd. or to a service center approved by B.A. International.

Use only original parts produced by the manufacturer.

If you have any questions, please contact your dental dealer or the manufacturer.


3 Technical description




3.1 Task

The BA 695 / BA 695FBA / BA 888FM turbine is used to power dental instruments with an FG shank.

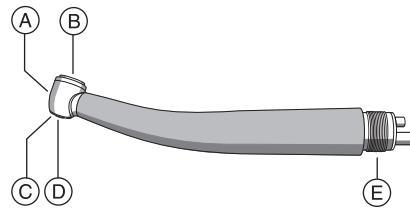3.2 Design of the BA ULTIMATE with fixed connection


A	Turbine head (in this case: Power)
B	Push button
C	Cooling spray outlet
D	Opening of chuck system
E	Fixed connection (in this case: Midwest/ISO 4-hole)

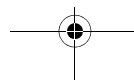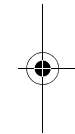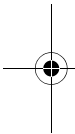3.3 Technical data

High-speed handpiece

		BA 695	BA 695FBA	BA 888FM
Available head sizes:				
	Power	x	x	x
	Mini	-	-	-
Light function		-	-	-
Back-suction stop in the head housing		x	x	-
Anti-retraction valve in the spray water area		-	x	-
Pushbutton cover clamping system		FG	FG	FG
No-load speed in rpm		400,000 ± 10%	400,000 ± 10%	400,000 ± 10%




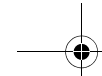
3 Technical description



Operating Instructions BA ULTIMATE



Friction grip bur

	Power
Shank standard	ISO 1797-1
Shank diameter in mm	1.59 - 1.60
Maximum total length in mm	25
Maximum working diameter in mm (ISO 2157)	2.1

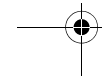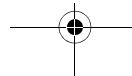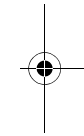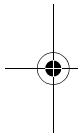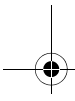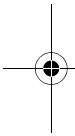
English

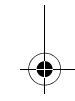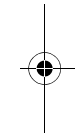Treatment center

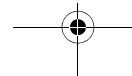	BA 695 / BA 695FBA	BA 888FM
Driving air pressure in bar (flowing, dry, and clean)	3.0	2.3
Driving air consumption in NI/min	52 ± 12	43 ± 4
Return air pressure in bar	< 0.3	< 0.15
Spray air pressure in bar	~ 2.7	~ 2.3
Spray air flow in NI/min	> 2.5	> 2.5
Spray water pressure in bar	~ 2.0	~ 2.0
Water content in spray ml/min	> 50	> 50

3.4 Turbine connection types available

	Connection type available for
	Midwest/ISO 4-hole fixed connection
	Borden/ISO 2/3-hole fixed connection

3.5 Production data

HSI HIBC code

H658 M 675LN 1 / \$ S 07 00001


A






B

C

A	Product label code (in this case: 675LN 1)
B	Year of manufacture (in this case: 2007)
C	Serial number (in this case: 00001)

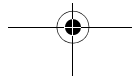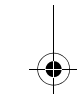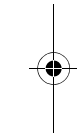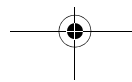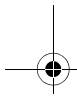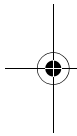65 56 083 D3560.201.10.03.02


4 Preparation

4.1 Initial start-up and longer breaks in use

- ✓ The turbine is only able to reach full power if indicated operating pressures are set [→ 13]. If necessary have the pressures checked by a service engineer with a star tester that is placed between the supply hose and the quick coupling.
- Sterilize the turbine and accessory parts prior to startup.
- Clean and maintain the turbine after longer breaks in use.

4.2 Prior to starting the work day



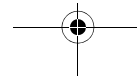
- Purge the water and air channels for 30 seconds.




4.3 Before each patient

1. Purge the water and air channels for 30 seconds.
2. Attach the turbine.
3. Insert the bur instrument.
4. Set a sufficient amount of cooling water (> 50 ml/min) [→ 20].
5. Use filtered water only (< 50 μ m).
6. Check the nozzles for blockages and lime deposits, for example, and clean the nozzles if necessary.

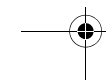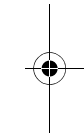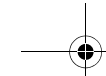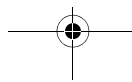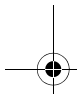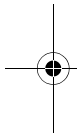⚠ CAUTION! Insufficient cooling leads to overheating of the preparation site and damage to the tooth substance. Ensure that the water content is > 50 ml/min.

5 Operation

NOTICE! Use only burs and diamond polishers that are sharp and undamaged. Use clean burs and diamond polishers to avoid dirt in the clamping system.

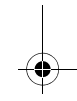
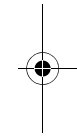
⚠ CAUTION! A loose or partially removed bur instrument can detach itself from the head or break off. This may cause injury! Therefore, use the turbine only if the bur instrument is inserted at least 10 mm and is clamped securely in place.

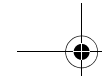
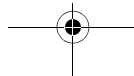
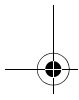
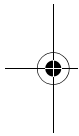
⚠ CAUTION! Insufficient cooling leads to overheating of the preparation site and damage to the tooth substance. Ensure that the water content is > 50ml/min.

⚠ CAUTION! Do not pull the patient's cheek back with the handpiece while the turbine is rotating. This would actuate the pushbutton, thus creating a risk of burning the patient's oral mucosa.

English







IMPORTANT: For application details and operating data of the different preparation tools, please refer to the information provided by the manufacturer.

5.1 Adjusting the cooling spray

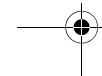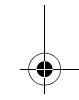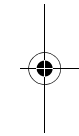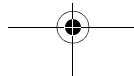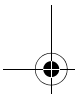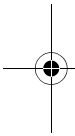- Adjust the cooling spray for the highspeed handpiece with fixed connection at the treatment unit.

6 Follow-up

6.1 After each treatment session

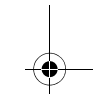
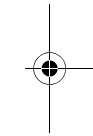
NOTICE! Condition immediately, or at the latest, one hour after treatment.

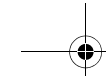
- ✓ The bur instrument must not be moving.
- ✓ Wear appropriate protective clothing.

1. Purge the water and air channels on the treatment center for 30 seconds.
2. Remove the bur instrument with tweezers.
3. Pre-disinfect directly at the treatment center [→ 23].
4. Remove the turbine.
5. Transport the turbine to the hygiene room in a suitable transport container.

English

6 Follow-up

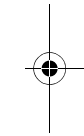
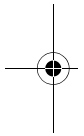
Operating Instructions BA ULTIMATE

- 6. Conduct automatic conditioning. Manual conditioning [→ 26] is possible in exceptional cases if the national/local regulations are followed.
- 7. Spray the turbine [→ 28].
- 8. Sterilize the turbine and the accessories [→ 32].



6.2 At the end of the work day



- Spray the turbine [→ 28].

22

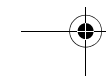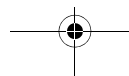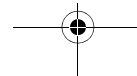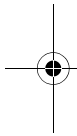65 56 083 D3560.201.10.03.02

7 Conditioning

7.1 Conduct pre-disinfection

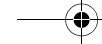
- ✓ Wear appropriate protective clothing.
 - ✓ All disinfectants must be approved in your country and have proven bactericidal, fungicidal and virucidal properties. Only use disinfectants with **no** protein-fixing properties.
1. Spray the surface with disinfectant.
 2. Wipe the disinfectant off with a cloth.
- ↪ For further processing, the turbine should be dry and free of residue.

In USA and Canada use **only** following agents:

- CAVICIDE®
- CAVIWIPES™


Please observe the manufacturer's instructions for using instrument disinfectants.

7 Conditioning

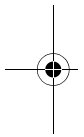Operating Instructions BA ULTIMATE

24

7.2 Automated cleaning and disinfecting

BA 695 / BA 695FBA / BA 888FM can also be cleaned and disinfected in suitable cleaning and disinfection equipment.

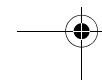
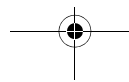
The cleaning and disinfection equipment used must be approved by its manufacturer for the cleaning and disinfection of dental instruments and comply with EN ISO 15883-1 (e.g. 95°C (203°F) and 10 min. holding time).

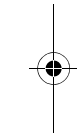
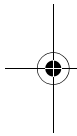
For further details, refer to the operating instructions supplied with the unit.

✓ BA 695 / BA 695FBA / BA 888FM is conditioned with a cleaning and disinfection device.

1. Check whether BA 695 / BA 695FBA / BA 888FM is clean after conditioning under good lighting (min. 500 lux) and color rendering index (min. 80 Ra).
2. If they are still dirty, repeat the process.


65 56 083 D3560.201.10.03.02




↳ For further processing, BA 695 / BA 695FBA / BA 888FM should be dry and free of residue.

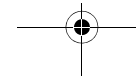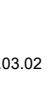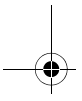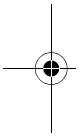3. Blow BA 695 / BA 695FBA / BA 888FM out with max. 3 bar.
4. Maintain mechanical parts manually [→ 28].
5. Maintain the push button chuck manually [→ 31].
6. Pack BA 695 / BA 695FBA / BA 888FM in packaging material suitable for sterilization and storage. e.g. paper/plastic composite packaging.
7. Conduct sterilization [→ 32].

7.3 Manual cleaning and disinfection

IMPORTANT: Manual conditioning is possible in exceptional cases if the national/regional regulations are followed. The national/regional regulations are to be checked before.

NOTICE! Condition immediately, or at the latest, one hour after treatment.

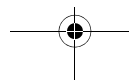
NOTICE! Never clean in an ultrasound bath!

NOTICE! Never immerse in disinfectant solution!

NOTICE! Using sprays from other manufacturers can reduce the product's service life. Only use BA Ultimate Spray.


IMPORTANT: Use a soft, clean, and disinfected brush for cleaning.

- ✓ Wear appropriate protective clothing.
- ✓ All disinfectants must be approved in your country and have proven bactericidal, fungicidal and virucidal properties. Only use disinfectants with **no** protein-fixing properties.


Operating Instructions BA ULTIMATE


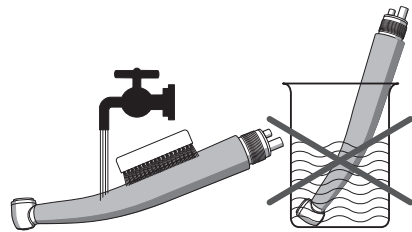
1. Brush the turbine under running water (< 38 °C, < 100 °F, at least drinking water quality) and good lighting (min. 500 lux) and color rendering index (min. 80 Ra) until no more dirt can be seen, for at least 10 seconds.
2. Flush the drive channels with spray.
3. Clean and disinfect spray channels with suitable agents and adapters according to manufacturer's instructions.
4. Blow spray channels with suitable adapters out with 2.5 - 3 bar until no more moisture displaces, but at least 10 seconds.
5. Conduct thermal disinfection or unwrapped steam sterilization.
6. Maintain mechanical parts manually [→ 28].
7. Maintain the push button chuck manually [→ 31].
8. Pack the turbine in packaging material suitable for sterilization and storage. e.g. paper/plastic composite packaging.
9. Sterilize the adapter [→ 32].

7 Conditioning

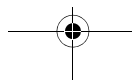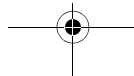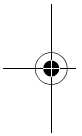English




7.4 Manual maintenance

7.4.1 Maintenance of mechanical parts


Intervals

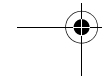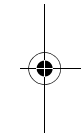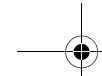- At least every noon and evening
- Prior to each sterilization
- After every thermal disinfection without integrated maintenance
- During a longer preparation (> 10 minutes)




Operating Instructions BA ULTIMATE


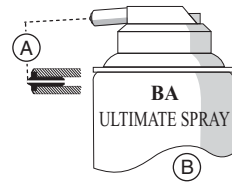
Required accessories


- A Black nozzle insert
- B BA Ultimate Spray

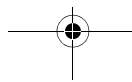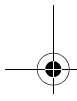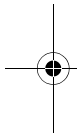NOTICE! Using sprays from other manufacturers can reduce the product's service life. Only use BA Ultimate Spray.




English

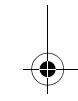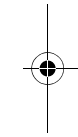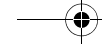7 Conditioning

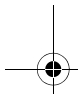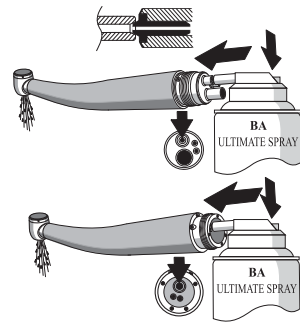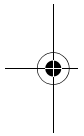30

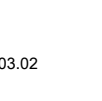
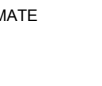
Process



✓ The black nozzle insert is in the spray can tube.

1. Take the highspeed handpiece off of the supply hose.
2. Insert the nozzle of the spray can in the driving air tube. Hold the highspeed handpiece securely with a cloth when doing this.
3. Spray into the turbine for 1-2 seconds.
IMPORTANT: Hold the spray can upright.
4. Wipe any spray that comes out with a disinfection cloth.
5. Repeat the process until the spray is clear.

Operating Instructions BA ULTIMATE

65 56 083 D3560.201.10.03.02

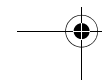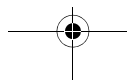7.4.2 Care of the push button chuck

To remove deposits and ensure proper functioning of the clamping system, the pushbutton chuck must be maintained with BA Ultimate Spray.

Interval

- At least once a week

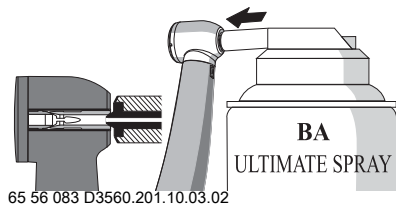
Required accessories

BA Ultimate Spray

Procedure

✓ The turbine is cleaned and disinfected.

1. Press the turbine head with the chuck firmly against the spray can nozzle.
2. Spray the chuck for 1 - 2 seconds.
IMPORTANT: Hold the spray can upright.
3. Wipe any spray that comes out with a disinfection cloth.


65 56 083 D3560.201.10.03.02

7 Conditioning

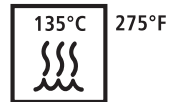Operating Instructions BA ULTIMATE


7.5 Sterilizing

- ✓ The turbine is cleaned and disinfected.
- ✓ Turbine maintenance is complete.
- ✓ If necessary, the turbine can be sterilized in packaging suitable for sterilization and storage, e.g. paper/plastic composite packaging or container.
- Sterilize the turbine in the steam sterilizer with saturated water vapor.

Overpressure: 2.04 bar (29.59 psi)


Temperature: 135°C (275° F)



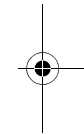
Holding time: >3 min.

32

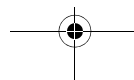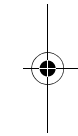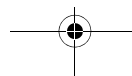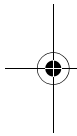65 56 083 D3560.201.10.03.02

Approved steam sterilizers are those that fulfill the requirements of either EN 13060, class B or EN 13060, class S and which are also suitable for the sterilization of highspeed handpieces.

NOTICE! Do not exceed 140°C (284°F), even during the drying phase.

After sterilizing

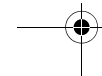
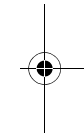
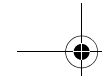
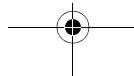
1. Remove the turbine from the steam sterilizer immediately.

⚠ CAUTION! The turbine will be hot. This may cause burns!

NOTICE! Do **not** immerse the turbine in cold water to speed up the cooling process. This will damage your turbine!

2. Store all turbines so that they are protected from contamination.

3. Sterilize again once the storage period has elapsed.

8 Maintenance

Elastomers, e.g. O-rings, must be replaced depending on their degree of wear.

8.1 Testing the FG clamping system

Interval

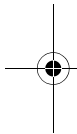
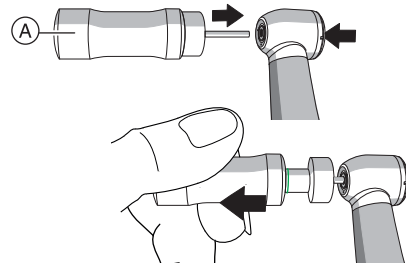
Test the FG clamping system at least once a month.

Required accessories

Chuck tester

Operating Instructions BA ULTIMATE

Process

✓ The expiry date of the chuck tester (A) has **not** elapsed.

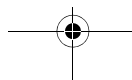
1. Insert the chuck tester into the FG clamping system.
2. Tighten the chuck tester until the marking ring appears (withdrawal force: 22 N)

Does the chuck tester slide out of the chuck before the marking ring appears?

⚠ CAUTION! The chuck is defective and the secure fit of the bur is not guaranteed. This may cause injury!

1. Do not use the product.
2. Have the clamping system replaced by a workshop authorized by B.A. International.

Tip: Record the time and result of the check for your own information.





6556083.book Seite 36 Dienstag, 22. November 2016 11:30 11

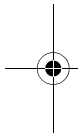
8 Maintenance


36

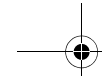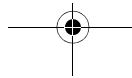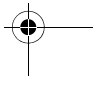Operating Instructions BA ULTIMATE

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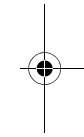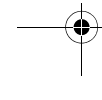
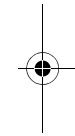
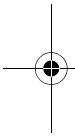
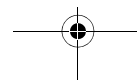
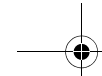
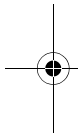
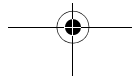
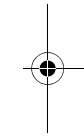
65 56 083 D3560.201.10.03.15

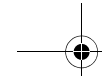Disposal

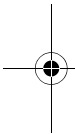
To the best of our current knowledge, the product does not contain any substances which are hazardous to the environment. The product must be sterilized before disposal. Please observe the applicable national regulations regarding disposal.

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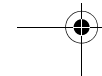
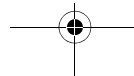
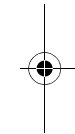
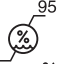
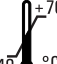
 Consult Instructions For Use	 Fragile, handle with care	 Keep dry	 Manufacturer	 Date of manufacture	 Can be sterilized	 Catalogue number	 Serial number	 ATTENTION	 10 95 % Relative humidity	 50 106 kPa Air pressure	 -40 +70 °C Tempe- rature
---	--	---	---	---	---	--	---	--	---	---	--

D3560.201.10.03.09 11.2016

Ä.-Nr.: 123 071

B.A. International Ltd.
Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton, NN2 6NE
United Kingdom

Made in Germany

65 56 083 D3560