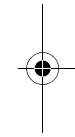
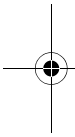
6453505_Titel.fm Seite 1 Donnerstag, 21. April 2016 5:12 17


BA ULTIMATE Kit

BA45 / BA175 / BA605 / BA606 / BA695FB / BA695FM











English / Español / Türkçe

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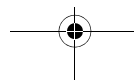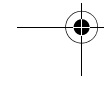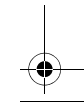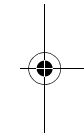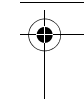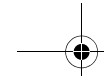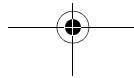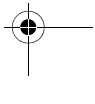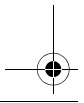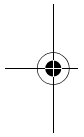
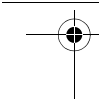
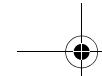
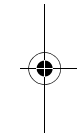
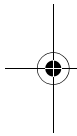
Operating Instructions




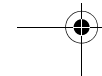
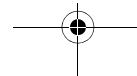
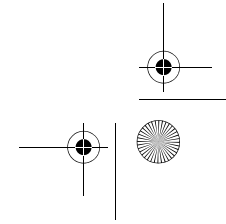
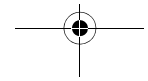
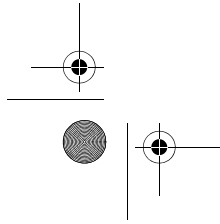
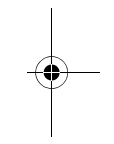
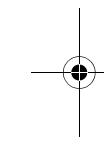
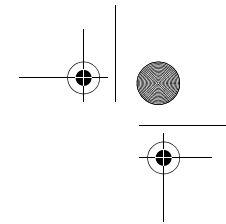
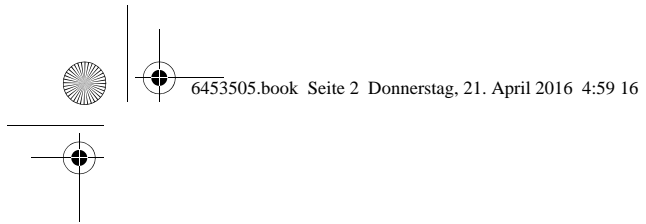
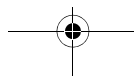
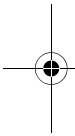
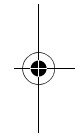
Table of Contents

1	Before you begin	6
1.1	Structure of the document	7
1.2	Service life	10
2	Safety instructions	11
3	Technical description	13
3.1	Task	13
3.2	BA ULTIMATE design	14
3.3	Technical data	17
3.4	Turbine connection types available	22
3.5	Production data	22

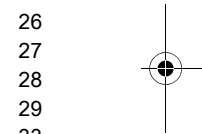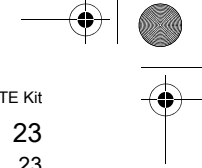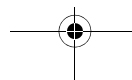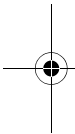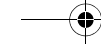6
7
10
11
13
13
14
17
22
22
3




4	Preparation	23
4.1	Initial start-up and longer breaks in use	23
4.2	Prior to starting the work day.....	23
4.3	Before each patient	24
5	Operation	25
5.1	Connect motor to the supply hose.....	26
5.2	Replacing the instrument.....	27
5.3	Connect turbine to the supply hose.....	28
5.4	Inserting and removing burr instruments.....	29
5.5	Inserting and removing the disposable prophylaxis head	33
5.6	Attaching and detaching the spray clip.....	35
6	Follow-up	36
6.1	After each treatment session.....	36
6.2	At the end of the work day.....	36



Operating Instructions BA ULTIMATE Kit

7	Care and maintenance	37
7.1	Spraying the mechanical parts	37
7.2	Care of the push button chuck	43
7.3	Manually cleaning and disinfecting the surface	44
7.4	Automated cleaning and disinfecting	46
7.5	Sterilizing	48
7.6	Cleaning the cooling spray nozzle	49
7.7	Testing the FG clamping system	50
7.8	Replacing the sealing washer	52
8	Storage and transport conditions	53

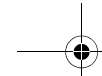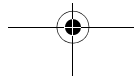
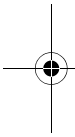
English

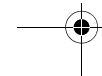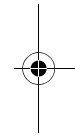
1 Before you begin ...

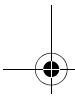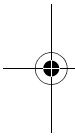Intended use

6

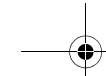Operating Instructions BA ULTIMATE Kit

1 Before you begin ...

BA ULTIMATE is used to treat dental infections and injuries.

The treatment involves the rotary processing of hard and soft tooth substance, bones, and dental prostheses (crowns, inlays, bridges, etc.).

The BA ULTIMATE instrument is used for prophylaxis.

BA ULTIMATE complies with state-of-the-art technical regulations.

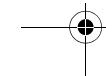
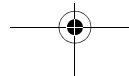
The instrument complies with the ISO 14457 standard.

The turbine complies with the ISO 14457 and ISO 9168 standards.

The motor complies with the ISO 14457 and ISO 9168 standards.

1. Read the operating instructions prior to using BA ULTIMATE.
2. Only use BA ULTIMATE for the applications described in the operating instructions.
3. Observe the applicable hygiene standards, occupational safety regulations and accident prevention measures for BA ULTIMATE.


64 53 505 D3551.201.04.02.02

Operating Instructions BA ULTIMATE Kit

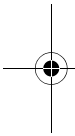Target group



Warnings


Instructions for use

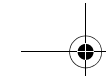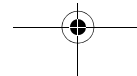64 53 505 D3551.201.04.02.02


1 Before you begin ...

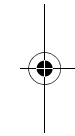English

7

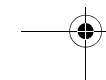BA ULTIMATE is intended only for use by trained dental personnel in dental practices and laboratories.

1.1 Structure of the document

1.1.1 Labeling of information

➤ To prevent injuries, please observe all warnings.

Warnings are labeled as follows:

⚠ **DANGER!** indicates a danger **leading** to death or serious injury if not avoided.

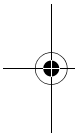
⚠ **WARNING!** indicates a danger that **may lead** to death or serious injury if not avoided.

⚠ **CAUTION!** indicates a danger that **may lead** to injury if not avoided.



➤ To prevent material damage and additional expenses, please observe all instructions for use.

1 Before you begin ...

8

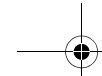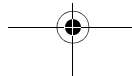
Operating Instructions BA ULTIMATE Kit

Instructions for use are labeled as follows:


NOTICE! indicates measures for the prevention of material damage.






IMPORTANT: indicates information on the avoidance of additional expenses and other important information.

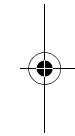Tip: indicates information for facilitating work.

64 53 505 D3551.201.04.02.02

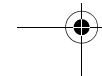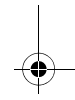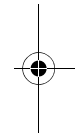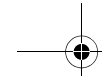
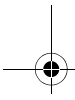
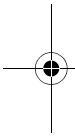
1.1.2 Formats and symbols

The formats and symbols used in this document have the following meaning:

<ul style="list-style-type: none"> ✓ Prerequisite 1. First action step 2. Second action step <p>or</p> <ul style="list-style-type: none"> > Alternative action <ul style="list-style-type: none"> ↵ Result 	Requests you to do something.
Use of formats and symbols [→ 9].	Identifies a reference to another text passage and specifies its page number.
• List	Identifies a list.

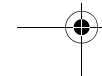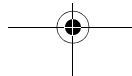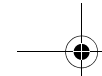1 Before you begin ...

Operating Instructions BA ULTIMATE Kit

1.1.3 Abbreviations/codes used


FG	Friction grip bur instrument
CA	Contra-angle handpiece bur instrument
HP	Straight handpiece bur instrument

1.2 Service life

When used as intended:



- Non-moving parts have a typical service life of 3 years
- Moving parts have a typical service life of 1.5 years

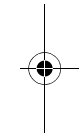No warranty claim can be inferred here, as wear may occur earlier or later than indicated above depending on use, frequency of sterilization, and frequency of maintenance.


10

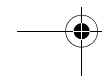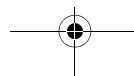64 53 505 D3551.201.04.02.02


Obligations of the user

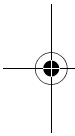Preventing the spread of infections and cross contamination

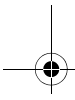Emitted cooling air



2 Safety instructions

- Use only fault-free materials that do **not** deviate from the specified data [→ 17].
- Protect yourself, patients, and others against any foreseeable dangers. To do this, follow the safety information.
- Use the equipment as intended.
- You should always keep these operating instructions within reach for further reference.

Prevent the spread of infections and cross contamination between patients, users, and third parties. Sterilize equipment after each patient. Take the appropriate hygiene measures, e.g. wear protective gloves.

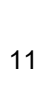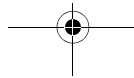The cooling air emitted by the coupling of the motor must have a flow rate of 1.5 - 10 NI/min.


2 Safety instructions

Instrument head overheating

Malfunction or damage

Repair

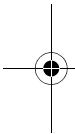Spare and accessory parts

If the instrument is defective, the area around the instrument head may heat up, thus creating a risk of burning the patient's oral mucosa.

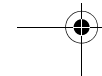
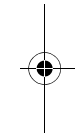
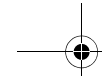
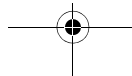
Discontinue use immediately in case of malfunction or damage. Damaged products may cause injury. Notify the dental depot or the manufacturer.

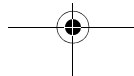
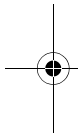
Do not attempt to repair BA ULTIMATE yourself.

The BA instrument is the result of a precise development and must **not** be disassembled by third parties. In the case of service or repair work, please send the instrument in an assembled state to B.A. International Ltd. or to a service center approved by B.A. International.

Use only original parts produced by the manufacturer.

If you have any questions, please contact your dental depot or the manufacturer.


3 Technical description

3.1 Task



The BA ULTIMATE instrument transmits the driving power and speed of the electric motor or air motor to the preparation tool.

The motor converts air to rotation and transfers rotation to the instrument.

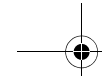The BA ULTIMATE turbine is used to power dental instruments with an FG shank.

English

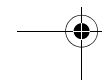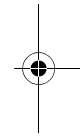3 Technical description

Operating Instructions BA ULTIMATE Kit


3.2 BA ULTIMATE design

Contra-angle handpiece

A	Instrument head
B	Push button
C	Opening for the clamping system


Straight handpiece

D	Opening for the clamping system
E	Center part of handpiece

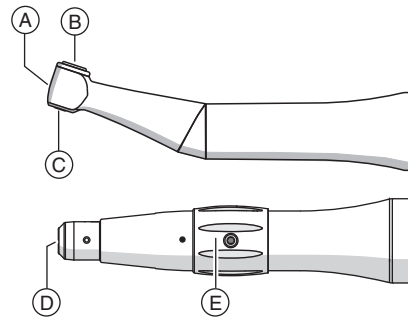Operating Instructions BA ULTIMATE Kit

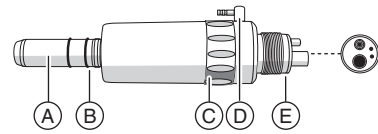Motor


A	Handpiece holder
B	O-ring 8.4 x 0.7
C	Drill direction switching ring
D	External spray connector
E	Coupling connection (here: Midwest/ISO 4-hole)

3 Technical description

English

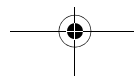
15





3 Technical description

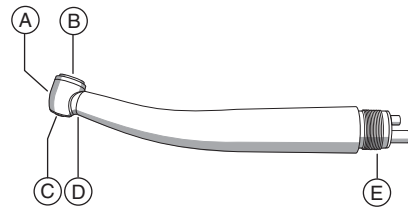Turbine


A	Turbine head
B	Push button
C	Opening of chuck system
D	Cooling spray outlet
E	Fixed connection (in this case: Midwest/ISO 4-hole)

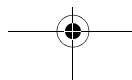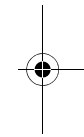
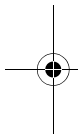
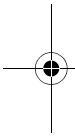
3.3 Technical data Instrument



	BA45	BA175
Transmission ratio (approx. specification)	1:1	1:1
Maximum motor speed in rpm	40000	40000
Maximum operating speed in rpm	~ 40000	~ 40000
Clamping system	CA	HP/CA
Instrument coupling	INTRAmatic®	INTRAmatic®


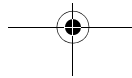
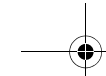
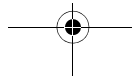
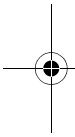
Air motor

	BA605 / BA606
Speed range in rpm	> 21 000
Torque in N/cm	~ 2.1
External spray function	x
Handpiece holder – ISO 3964	x

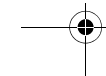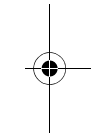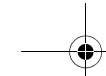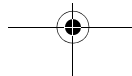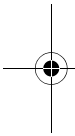English

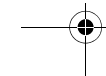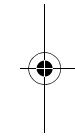Turbine


	BA 695
Light function	-
Back-suction stop in the head housing	x
Back suction stop in the spray water area	-
Pushbutton cover clamping system	FG
No-load speed in rpm	400 000 ± 10 %

Bur instruments


	FG	CA	HP
Shank diameter in mm	1.59 - 1.60	2.334 - 2.35	2.334 - 2.35
Maximum total length in mm	25	34	50
Maximum working diameter in mm	2.1	-	-
Standard	ISO 1797-1 ISO 2157	ISO 1797-1	ISO 1797-1

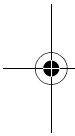
Dental treatment center

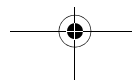	Motor	Turbine
Driving air pressure in bar (flowing, dry, and clean)	2.7	2.1 - 2.3
Return air pressure in bar	< 0.15	< 0.15
Air consumption in l/min	~ 55	43 ± 4
Spray air pressure in bar	2.3 ± 0.2	2.3 ± 0.2
Spray water pressure in bar	2.0 ± 0.2	2.0 ± 0.2
Recommended water content of spray	> 50 ml/min	> 50 ml/min

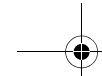3.4 Turbine connection types available

	Connection type available for
	Midwest/ISO 4-hole fixed connection
	Borden/ISO 2-hole fixed connection

3.5 Production data

HSI HIBC code



H658 M 675LN 1 / \$ S 07 00001



A

B

C

A	Product label code (in this case: 675LN 1)
B	Year of manufacture (in this case: 2007)
C	Serial number (in this case: 00001)

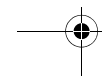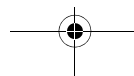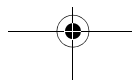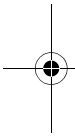4 Preparation

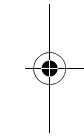
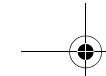
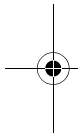
4.1 Initial start-up and longer breaks in use

- Sterilize BA ULTIMATE and accessories prior to startup.
- Clean and maintain BA ULTIMATE after longer breaks in use.

4.2 Prior to starting the work day

- Purge the water and air channels for 20-30 seconds.






4.3 Before each patient

1. Purge the water and air channels for 20-30 seconds.
2. Connect BA ULTIMATE.
3. Insert the burr instrument [→ 29].
4. Use a sufficient amount of cooling water (> 50 ml/min).
5. Use filtered water only (< 50 µm).
6. Check the nozzles for blockages and lime deposits, for example, and clean the nozzles if necessary.

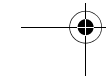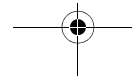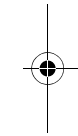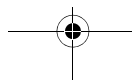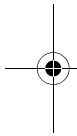⚠ CAUTION! Insufficient cooling leads to overheating of the preparation site and damage to the tooth substance. Ensure that the water content is > 50ml/min.




5 Operation

NOTICE! Use only burs and diamond polishers that are sharp and undamaged. Use clean burs and diamond polishers to avoid dirt in the clamping system.

⚠ CAUTION! A loose or partially removed bur instrument can detach itself from the head or break off. This may cause injury! Therefore, use BA ULTIMATE only if the bur instrument is inserted at least 10 mm and is clamped securely in place.

⚠ CAUTION! Insufficient cooling leads to overheating of the preparation site and damage to the tooth substance. Ensure that the water content is > 50ml/min.

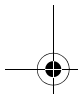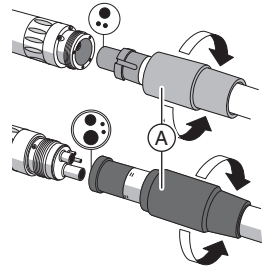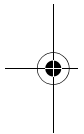⚠ CAUTION! Do not pull the patient's cheek back with the contra-angle handpiece while the motor is running. This would actuate the pushbutton, thus creating a risk of burning the patient's oral mucosa.

IMPORTANT: For application details and operating data of the different preparation tools, please refer to the information provided by the manufacturer.






5.1 Connect motor to the supply hose







⚠ CAUTION! Do not detach the motor from the supply hose during operation! This may cause injury!

1. Attach the motor to the supply hose, noting the various tube diameters.
2. Screw the screw cap (A) hand-tight. Hold the motor securely when doing this.

Does water leak out between the motor and the hose connection?

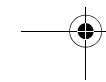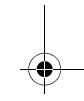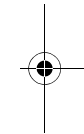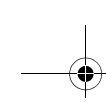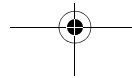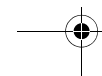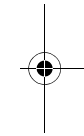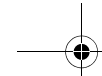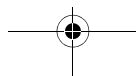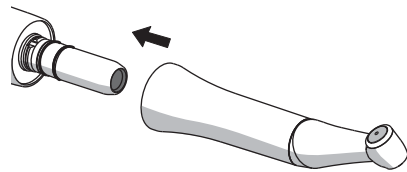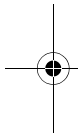1. Detach the motor from the supply hose.
2. Reconnect the motor to the supply hose. Make sure it is connected properly.

3. If water still leaks, replace the sealing washer [→ 52].

5.2 Replacing the instrument

⚠ CAUTION! The instrument should only be fitted or removed when the motor is at standstill.

Attaching the BA ULTIMATE

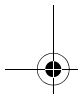
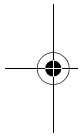
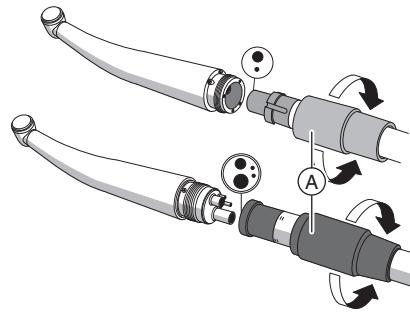
- ✓ The motor is at a standstill.
- Insert the instrument until it snaps into place.

Removing the BA ULTIMATE

- ✓ The motor is at a standstill.
- Pull off the instrument. Do not pull on the supply hose while doing this.

5 Operation

28

5.3 Connect turbine to the supply hose




Connecting the highspeed handpiece




1. Attach the highspeed handpiece to the supply hose, observing the various tube diameters.
2. Screw the screw cap (A) hand-tight. Hold the highspeed handpiece securely when doing this.

Removing the highspeed handpiece

1. Unscrew and remove the screw cap.
2. Pull the highspeed handpiece off of the supply hose.

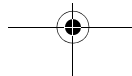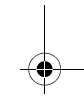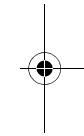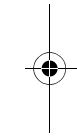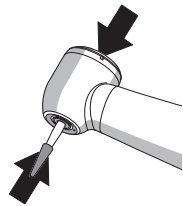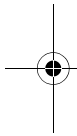Operating Instructions BA ULTIMATE Kit

5.4 Inserting and removing bur instruments

IMPORTANT: Check the push button to make sure it moves freely!

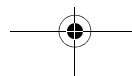
NOTICE! Use only burs and diamond polishers that are sharp and undamaged. Use clean burs and diamond polishers to avoid dirt in the clamping system.

⚠ CAUTION! A loose or partially removed bur instrument can detach itself from the head or break off. This may cause injury! Therefore, use BA ULTIMATE only if the bur instrument is inserted at least 10 mm and is clamped securely in place.

Inserting the friction grip bur instrument




✓ The turbine is at a standstill.

1. Press the button and slide the bur instrument in until it reaches the stop.
2. Pull on the bur instrument to check that it is firmly attached.




5 Operation

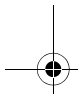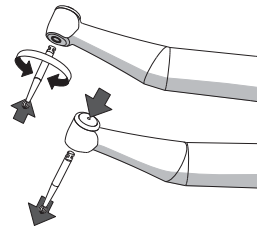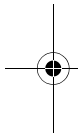30

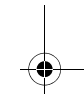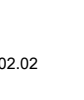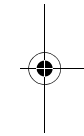Operating Instructions BA ULTIMATE Kit

Removing the friction grip bur instrument

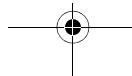
- ✓ The bur instrument must not be moving.
- Press the button and remove the bur instrument.

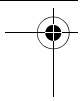
Inserting the contra-angle handpiece bur instrument

- ✓ The motor is at a standstill.
- 1. Insert the bur instrument without pushing the button.
- 2. Turn the bur instrument back and forth gently until it snaps into place.
- 3. Pull and turn the bur instrument to check that it is firmly seated.



Removing the contra-angle handpiece bur instrument

- ✓ The bur instrument must not be moving.
- Press the button and remove the bur instrument.

Operating Instructions BA ULTIMATE Kit

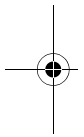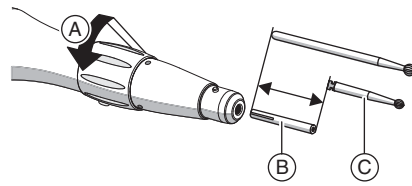
Inserting the straight handpiece bur instrument








✓ The motor is at a standstill.

1. Turn the center part of the handpiece in the direction of the arrow (A) until it reaches the stop.
2. If you are using a **contra-angle handpiece bur instrument (C)**: insert the pin (B) into the handpiece, slit end first. The pin compensates for the difference in length between straight and contra-angle handpiece bur instruments.
3. Insert the bur instrument until it reaches the stop.

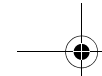5 Operation




English

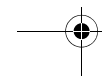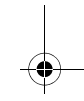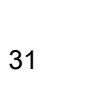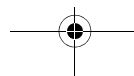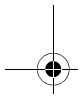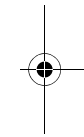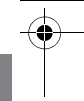5 Operation

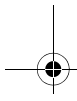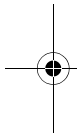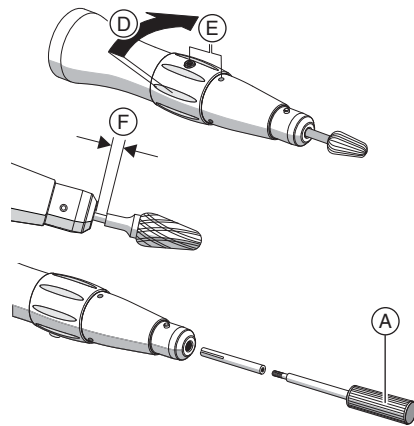32

4. Clamp the bur instrument by turning the center part of the handpiece in direction (D) until it reaches the stop.

↳ When the screw and the marks (E) line up with one another, the bur instrument is clamped.

↳ When using handpiece bur instruments with a reduced shank: always make sure that the reduced shank does not come into contact with the opening for the chuck (F).

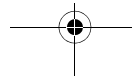
5. Pull and turn the bur instrument to check that it is firmly seated.

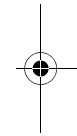
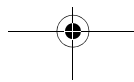
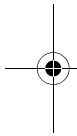
Removing the pin

1. Screw in the supplied tool (A).

2. Open the clamping system.

3. Pull out the pin.

5.5 Inserting and removing the disposable prophylaxis head

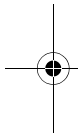
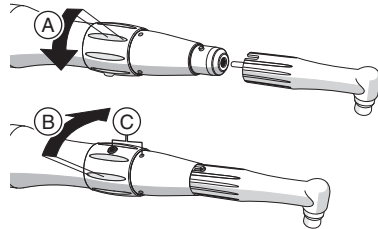
The BA ULTIMATE handpiece fits disposable prophylaxis heads (e.g., Doriot type made of plastic).

⚠ CAUTION! The handpiece must only be operated with the clamping system closed.

IMPORTANT: Reduce the maximum operating speed of the handpiece according to the manufacturer's specifications on the dental unit

5 Operation

Inserting a disposable prophylaxis head

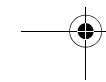
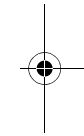
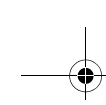
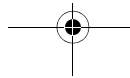
✓ The motor is at a standstill.

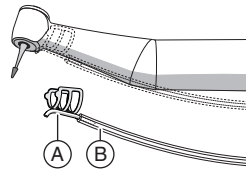
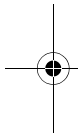
1. Turn the center part of the handpiece in the direction of the arrow (A) until it reaches the stop.
2. Insert the drive shaft of the disposable prophylaxis head into the handpiece until it reaches the stop.
3. Close the clamping system by turning the center part of the handpiece in the direction of the arrow (B) until it reaches the stop.
 - ↳ When the screw and the mark (C) are lined up with one another, the disposable prophylaxis head is clamped.

Removing the disposable prophylaxis head

✓ The motor is at a standstill.

1. Open the clamping system.
2. Remove the disposable prophylaxis head.





5.6 Attaching and detaching the spray clip



Attaching the spray clip

1. Attach the silicone hose (B) to the small tube on the spray clip (A).
2. Plug the spray clip onto the instrument.

Detaching the spray clip

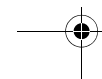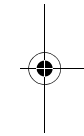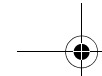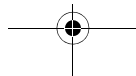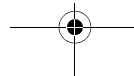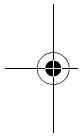1. Remove the spray clip from the instrument.
2. Flush out the spray clip and silicone hose with clear water.




6 Follow-up



6.1 After each treatment session

1. Clean and disinfect BA ULTIMATE and accessories.
2. Care BA ULTIMATE with spray [→ 37].
3. Sterilize BA ULTIMATE and the accessories [→ 48].

6.2 At the end of the work day

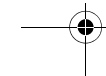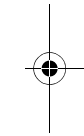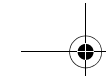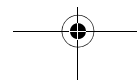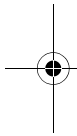- Care BA ULTIMATE with Spray [→ 37].

7 Care and maintenance

Elastomers, e.g. O-rings, must be replaced depending on their degree of wear.

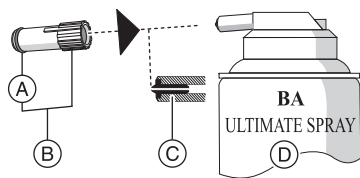
Check the clamping system annually for safety reasons.

7.1 Spraying the mechanical parts

Intervals

- At least every noon and evening
- Prior to each sterilization
- After each thermodisinfection
- During a longer preparation (> 10 minutes)

7 Care and maintenance

Required accessories

- A O-ring for spray adapter
- B Spray adapter
- C Black nozzle insert
- D BA Ultimate Spray

NOTICE! Using sprays from other manufacturers can reduce the product's service life. Only use BA Ultimate Spray.

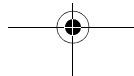
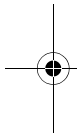
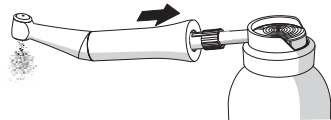
Process

Instrument




- ✓ The motor is at a standstill.
 - ✓ The O-ring on the spray adapter must be intact.
1. Remove the burr instrument [→ 29].

Operating Instructions BA ULTIMATE Kit

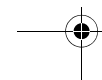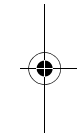
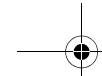
2. Pull the instrument off the motor [→ 27].
 3. Fit the spray adapter onto the nozzle of the spray can.
 4. Insert the instrument until it snaps in place and hold it.
 5. Spray the instrument for 1 - 2 seconds.
- IMPORTANT:** Hold the spray can upright.




7 Care and maintenance

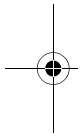
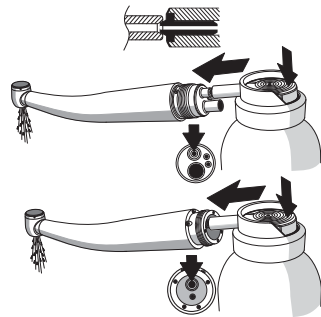
Highspeed handpiece with fixed connection

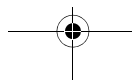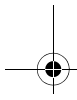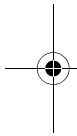- ✓ The turbine is at a standstill.
- ✓ The black nozzle insert is in the spray can tube.
- 1. Remove the bur instrument.
- 2. Take the highspeed handpiece off of the supply hose.
- 3. Insert the nozzle of the spray can in the driving air tube. Hold the highspeed handpiece securely with a cloth when doing this.
- 4. Spray into the turbine for 1-2 seconds.
IMPORTANT: Hold the spray can upright.

English


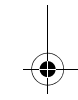
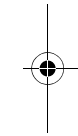
Check

1. Check whether the fluid coming out of the head is clean.
2. If the fluid is dirty, repeat the process.

Is the fluid still not clean?

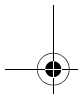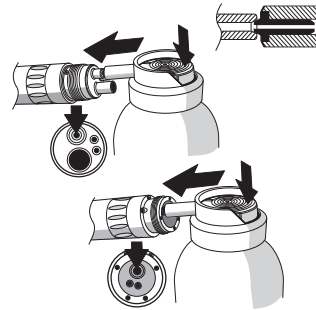
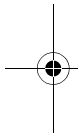
1. Connect BA ULTIMATE and let the drive run briefly. In this way, the spray can be distributed more effectively.
2. **⚠ CAUTION!** Stop BA ULTIMATE. Wipe off any leaking oil with a dry cloth.
3. Repeat the spraying procedure.



Air motor

- ✓ The motor is at a standstill.
- ✓ The instrument is removed [→ 27].
- ✓ The black nozzle insert is in the spray can tube.

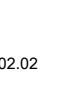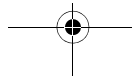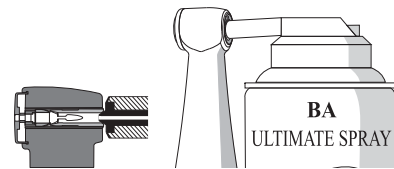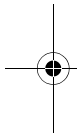1. Detach the motor from the supply hose [→ 26].
2. Insert the nozzle of the spray can in the driving air tube. Hold the motor with a cloth.
3. Spray into the motor for 1-2 seconds.
IMPORTANT: Hold the spray can upright.

64 53 505 D3551.201.04.02.02

7.2 Care of the push button chuck

To remove deposits and ensure proper functioning of the clamping system, the pushbutton chuck must be maintained with BA Ultimate Spray.

Interval

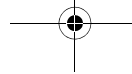
Carry out maintenance work on the pushbutton chuck at least once a week.

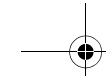
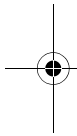
Required accessories

BA Ultimate Spray

Process

1. Press the contra-angle handpiece head/turbine head with the chuck firmly against the spray can nozzle.
2. Spray the chuck for 1 - 2 seconds.
IMPORTANT: Hold the spray can upright.

3. Wipe off any leaking protective oil with a lint-free cloth.

7.3 Manually cleaning and disinfecting the surface

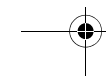
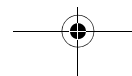
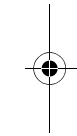
✓ Wear appropriate protective clothing.

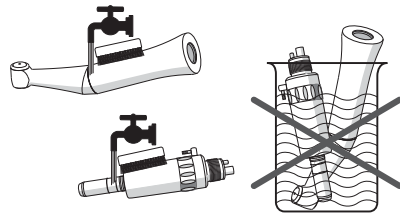
1. Remove the burr instrument [→ 29].

2. Use automated processing when possible.

NOTICE! Disinfect and clean the motor **only** by hand.

IMPORTANT: Manual cleaning must always be combined with disinfection.

7.3.1 Cleaning the outer surface

NOTICE! Never clean in an ultrasonic bath!

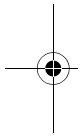
1. Clean BA ULTIMATE by brushing it off under running water (< 38°C, < 100°F, at least drinking water quality).
2. Blow BA ULTIMATE out with max. 3 bar.
3. Spray BA ULTIMATE spraying the mechanical parts [→ 37].

7.3.2 Disinfecting the surface

NOTICE! Never immerse in disinfectants!

- ✓ The disinfectants must be approved in your country and have proven bactericidal, fungicidal and virucidal properties.
1. Spray the surface with disinfectant.

2. Wipe the disinfectant away with a cloth.

In the USA and Canada, for example, you can use:

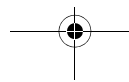
- CAVICIDE®
- CAVIWIPES™

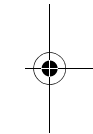
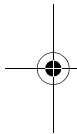
Please observe the manufacturer's instructions for using instrument disinfectants.

7.4 Automated cleaning and disinfecting

NOTICE! Clean the motor mechanically **only** from the outside.

BA ULTIMATE can also be cleaned and disinfected in suitable cleaning and disinfection equipment.




The cleaning and disinfection equipment used must be approved by its manufacturer for the cleaning and disinfection of dental instruments and comply with EN ISO 15883-1 (e.g. 95°C (203°F) and 10 min. holding time).

For further details, refer to the operating instructions supplied with the unit.

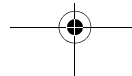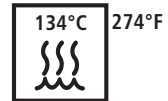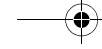1. Check whether BA ULTIMATE is clean after processing.
2. If it is still dirty, repeat the process.
 - ↳ For further processing, BA ULTIMATE should be dry and free of residue.
3. Blow BA ULTIMATE out with max. 3 bar.
4. Manual care of mechanical parts [→ 37].
5. Manual care of the push button chuck [→ 43].

7.5 Sterilizing

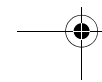
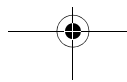
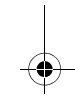
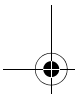
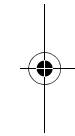
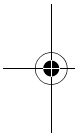
1. Clean and disinfect BA ULTIMATE.
2. Care BA ULTIMATE with spray [→ 37].
3. Sterilize BA ULTIMATE in the steam sterilizer with saturated water vapor.

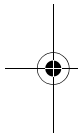
Saturated water vapor: Temperature: 134°C (274°F)
Overpressure: 2.04 bar (29.59 psi)

Holding time: 3 min

Steam sterilizers that meet the requirements of EN 13060, class B or S, and are also suitable for the sterilization of straight/contra-angle handpieces are approved.

NOTICE! Do not exceed 140°C (284°F), even during the drying phase.

BA ULTIMATE can be sterilized in packaging that is suitable for sterilization and storage, e.g. paper/plastic composite packaging.

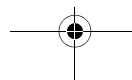
After sterilizing

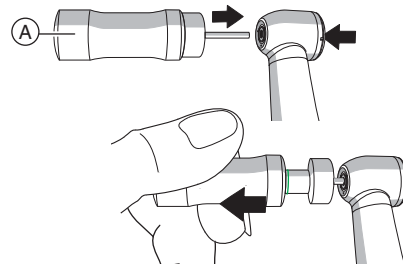
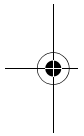
1. Remove BA ULTIMATE from the steam sterilizer immediately.
⚠ CAUTION! BA ULTIMATE is hot.
NOTICE! Do **not** try to accelerate the cool-down process by placing it in cold water. This will damage your BA ULTIMATE!
2. Store BA ULTIMATE so that it is protected against contamination.
3. Sterilize again once the storage period has elapsed.

7.6 Cleaning the cooling spray nozzle

If your tap water is very hard, calcareous deposits can constrict or completely block the cooling spray nozzle.

1. Carefully clean the nozzle opening on a regular basis by running a cleaning wire through it.

7.7 Testing the FG clamping system

Interval

Test the FG clamping system at least once a month.

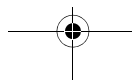
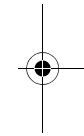
Required accessories

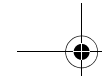
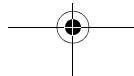
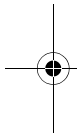
Chuck tester

Process



✓ The expiry date of the chuck tester (A) has **not** elapsed.

1. Insert the chuck tester into the FG clamping system [→ 29].
2. Tighten the chuck tester until the marking ring appears (withdrawal force: 22 N)

English

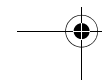
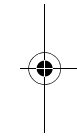
Does the chuck tester slide out of the chuck before the marking ring appears?

⚠ CAUTION! The chuck is defective and the secure fit of the bur instrument is not guaranteed. This may cause injury!

1. Do not use the product.
2. Have the clamping system replaced by a workshop authorized by BA International.





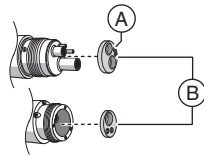
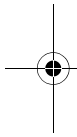
Tip: Record the time and result of the check for your own information.



7.8 Replacing the sealing washer



Water leaks between the motor/the turbine and the hose connection.

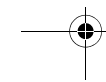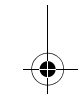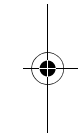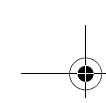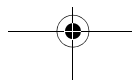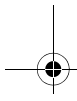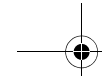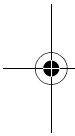1. Disconnect the motor/the turbine from the supply hose.
2. Use a probe etc. to remove the defective sealing washer (B) from the rear of the motor/the turbine.
3. Attach the new sealing washer, observing the position of the tubes and contact pins and press it to the stop.
IMPORTANT: The bead (A) on the sealing washer faces the supply hose.

English

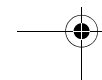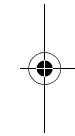
8 Storage and transport conditions

Protect from moisture

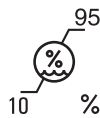Sensitive contents


95
10 %

Relative humidity

106
50 kPa

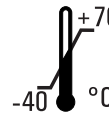
Air pressure

+70
-40 °C











Temperature

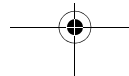After a severe change in temperature, allow sufficient time for acclimatization.

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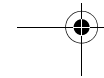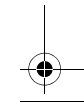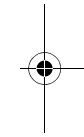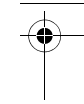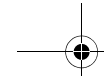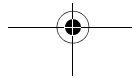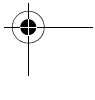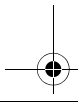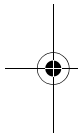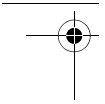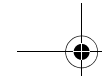
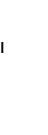
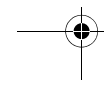
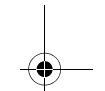
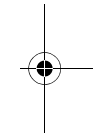
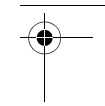
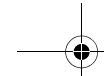
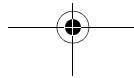
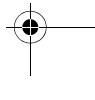
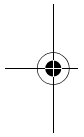
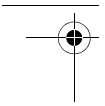
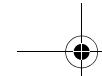
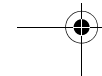
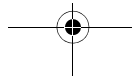
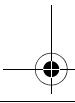
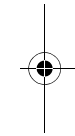
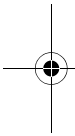
Eliminación

İmha etme

To the best of our current knowledge, the product does not contain any substances which are hazardous to the environment. The product must be sterilized before disposal. Please observe the applicable national regulations regarding disposal.

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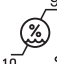
 Consult Instructions For Use	 Fragile, handle with care	 Keep dry	 Manufacturer	 Date of manufacture	 Can be sterilized	 Catalogue number	 Serial number	 ATTENTION	 10 95 % Relative humidity	 50 106 kPa Air pressure	 -40 +70 °C Tempe- rature
---	--	---	---	---	---	--	---	--	---	---	--

D3551.201.04.02.09 07.2016

Ä.-Nr.: 121 960

B.A. International Ltd.
Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton, NN2 6NE
United Kingdom

Made in Germany

64 53 505 D3551