BA ULTIMATE

Operation Manual for:


- BA604ES
BA641065

- BA602ES
BA641085


CE 0197 **Made in Japan**

OM-MK154E 002


Please read this operation manual carefully before use, and file for future reference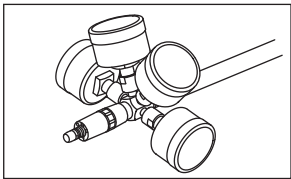


1

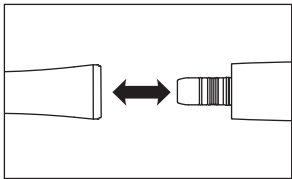
2

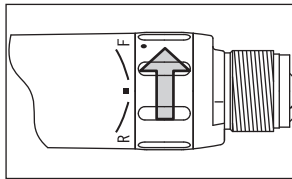
3

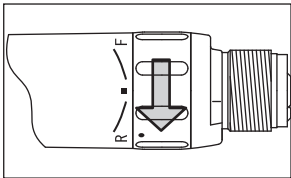
4

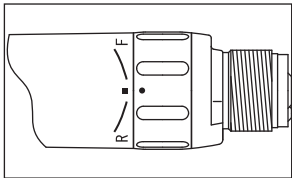
5

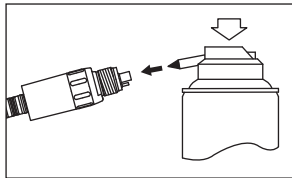6

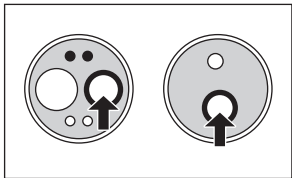7

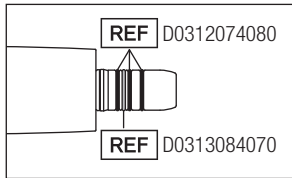8

1. User and Indications for Use

User: Qualified Professionals (Dentist / Dental hygienist)

Indications for Use:

The air motor is intended to drive a dental handpiece or contra-angle handpiece, for use by qualified personnel in the performance of dental restoration, prophylaxis, endodontic procedures.

2. Precautions for handling and operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
 WARNING	Hazard that could result in serious injury or damage to the device if the safety instructions are not followed.
 CAUTION	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not followed.
NOTICE	General product specification information highlighted to avoid product malfunction and performance reduction.

CAUTION

- Read this Operation Manual before use to fully understand the product functions and file for future reference.
- When operating the product always consider the safety of the patient.
- This product does not consider patient's age (except infants), gender, weight or nationality.
- This product does not consider operator's age (mature person), height, weight, gender, or nationality.

- Users are responsible for the operational control, maintenance and continual inspection of this product.
- Do not attempt to disassemble the product or tamper with the mechanism except as recommend by B.A. International in this Operation Manual.
- Do not allow any impact on to the product. Do not drop the product.
- Operators and all others in the area must wear eye protection and a mask when operating this motor.
- Should the product function abnormally, cease operation immediately and contact your Authorized B.A. International Dealer.
- Do not use high acid water or sterilizing solutions to wipe, immerse or clean the product.
- The products are delivered in a non-sterile condition and must be autoclaved prior to use.
- Perform regular function and maintenance checks.
- If the product is not used for a long period check it is functioning correctly before using on a patient.
- To avoid clinical downtime it is recommended that a spare be kept on hand in case of a breakdown during surgery.

3. Accessory List

Part Name	Quantity
Lubrication Oil	1

4. Setting of Air & Water Supply Pressure

Measure the supply pressure at the handpiece / hose connection point and set the pressure to the value specified on the specification table. (Fig. 1)

For Multi Gauge information refer to Option Parts List.

WARNING

- Do not exceed the optimum pressure specified on the specification table.

CAUTION

- Do not use air contaminated by dust, moisture and oil.

5. Connection & Disconnection of the Handpiece

5-1 Connection

- 1) Connect the handpiece direct to the motor. (Fig. 2)
- 2) Make sure the handpiece is firmly connected to the motor.

5-2 Disconnection

Hold the motor and the handpiece separately and pull them out straight.

CAUTION

- Disconnect the handpiece only after the motor has completely stopped rotating.

6. Check before treatment

Connect the handpiece to the motor and check for motor vibration, noise and overheating. If any abnormalities are found do not use the motor and contact your Authorized B.A. International Dealer.

7. Forward/Reverse Rotation

- 1) Forward Rotation : Turn the FWD/REV Change Ring to "F". (Fig. 3)
- 2) Reverse Rotation : Turn the FWD/REV Change Ring to "R". (Fig. 4)
- 3) When the FWD/REV Change Ring is positioned at the middle of "F" and "R" , the drive air is interrupted and the motor does not rotate. (Fig. 5)

8. Post-use Maintenance

After each patient maintain the product as follows.

WARNING

- Immediately after a treatment, the handpiece should be cleaned, lubricated and sterilized. Failure to properly maintain the handpiece may lead to overheating, causing burn injuries or product failure.

CAUTION

- Do not use the following fluids to wipe, immerse or clean the product; strong/super acid water, strong acid/alkaline chemicals, chlorine-containing solutions, solvents such as benzine or thinner.

8-1 Preparation

- 1) Wear a eye protection, a mask and gloves to avoid infection.
- 2) Disconnect the handpiece from the motor.
- 3) Remove the motor from the hose.

8-2 Cleaning

■ Manual Cleaning (External Surface)

- 1) Remove dirt and debris on the motor surface using a brush under running water (at 38°C or less). Do not use a wire brush.
- 2) Wipe off the moisture on the surface using absorbent cloth.

■ Automatic Cleaning (External and Internal Surfaces)

Use a thermo-disinfector in accordance with ISO 15883 (EN ISO 15883). Refer to the thermo-disinfector manual.

CAUTION

- After washing with thermo-disinfector and prior to lubrication, dry the product until all internal moisture is thoroughly removed. Thermo-disinfector moisture remaining inside the product could reduce the effect of lubrication and could cause corrosion inside of the product.

8-3 Disinfection

■ Manual Disinfection (External Surface)

Wipe the surface with cloth soaked in disinfectant solution such as alcohol.

■ Automatic Disinfection (External and Internal Surfaces)

Use a thermo-disinfector in accordance with ISO 15883 (EN ISO 15883). Refer to the thermo-disinfector manual.

8-4 Lubrication

■ BA ULTIMATE SPRAY

Apply BA ULTIMATE SPRAY every time after each use and/or before autoclaving.

- 1) Remove the motor from the hose.
- 2) Mount the arrow-head spray nozzle tip into the spray can port. (Fig. 6)
- 3) Insert the Spray Nozzle into the Drive Air Port of the motor. Hold the motor and spray for approximately 2-3 seconds. Apply lubricant until it expels from the exhaust air part for at least 2 seconds. (Fig. 7)

CAUTION

- Hold the spray can upright.
- Be sure to hold the handpiece firmly to prevent slipping out of hand by the spray pressure when lubricating.

■ Lubrication Oil

- 1) Remove the motor from the hose.
- 2) Apply 1-2 drops of lubrication oil into the Drive Air Port shown in the Fig. 7.
- 3) Connect the motor to the hose. Operate the motor for about 10 seconds.

8-5 Sterilization

Sterilize the product by autoclave sterilization. After each patient sterilize as noted below.

- 1) Insert into an autoclave pouch. Seal the pouch.
- 2) Autoclavable under the conditions below.
Autoclave for more than 15min. at 132°C, or 3min. at 134°C.
- 3) The handpiece should remain in the autoclave pouch until required for use.

CAUTION

- Do not autoclave the product with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments.
- Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product.
- To avoid product failure, do not use a sterilizer whose temperature exceeds 138°C during use including the dry cycle. The chamber temperature of some type of sterilizers may exceed 138°C. Contact the sterilizer manufacturer for detailed information.
- Keep the product in suitable atmospheric pressure, temperature, humidity, ventilation, and sunlight. The air should be free from dust, salt and sulphur.
- Do not touch the product immediately after autoclaving as it will be very hot and must remain in a sterile condition.
- Autoclave sterilization is recommended for the product. The validity of other sterilization methods is not confirmed.

NOTICE

- B.A. International recommends Class B sterilizers as stated in EN 13060.

9. Replacing the O-rings

Replace the O-rings if water is present in the exhaust air line. This is an indication of possible water leakage within the motor. ALWAYS change the complete set of O-rings.

- 1) Gently remove each O-ring by hand.
- 2) Insert the complete set of new O-rings in the correct grooves.

* Refer to Spare Parts List to identify the correct parts.

CAUTION

- Do not force the new replacement O-ring with excessive pressure.
- The blue O-Ring is thinner than the 3 black O-Rings. When inserting new O-rings, make sure they are inserted in the correct grooves as shown in the Fig. 8.

10. Periodical Maintenance Checks

Perform periodical maintenance checks every three months, referring to the check sheet below. If any abnormalities are found, contact your Authorized B.A. International Dealer.

Points to check	Details
Rotation	Rotate the motor and check for abnormalities such as abnormal rotation, vibration, noise, and overheating.

11. Symbol

This product is Autoclavable up to Max.135°C.

This product can be washed via Thermo Disinfector.

Conforms to CE European Directive of "Medical equipment directive 93/42/EEC."

Manufacturer.

Authorized representative in the European community.

Consult operation instructions.

12. Warranty

B.A. International products are warranted against manufacturing errors and defects in materials. B.A. International reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose or has been tampered with by unqualified personnel or has had non B.A. International parts installed. Replacement parts are available for seven years beyond discontinuation of the model.

13. Spare Parts List

Model	Order Code
O-Ring (Blue)	D0313084070
O-Ring (Black)	D0312074080

14. Option Parts List

Model	Order Code	Compatible Product
BA Multigauge for Midwest connections	BA08010030	BA604ES
Borden to Midwest Adaptor	TST790	BA602ES + BA Multigauge

15. Disposing product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled industrial wastes, to dispose the product for you.

16. Service and Repair

Important

This airmotor is a result of precision engineering and cannot be dismantled/repared by the third party. For service and repair, always send it to either BA International Ltd. or an accredited BA certified service centre without dismantling it.

17. Specifications

Model	BA604ES	BA602ES
Hose Connection Type	ISO 9168 Type 2 (Midwest 4 hole)	ISO 9168 Type1 (Borden 2 hole)
Handpiece Connection Type	E Type ISO 3964	
Max. Rotation Speed	19,800 - 24,200min ⁻¹ (0.25MPa)	
Breakdown Torque	1.5N·cm and more (0.25MPa)	
Drive Air Pressure	0.20 - 0.25MPa (2.0 - 2.5kgf/cm ²)	
Air Consumption	<66NL/min (0.25MPa)	
Water Pressure	0.10 - 0.25MPa (1.0 - 2.5kgf/cm ²)	
Chip Air Pressure	0.15 - 0.25MPa (1.5 - 2.5kgf/cm ²)	-
Coolant Water Supply	50mL/min and more (0.25MPa)	
Coolant Air Supply	1.5NL/min and more (0.25MPa)	-
Water Spray Type	Internal Spray	
Use Environment	Temperature: 10 - 35°C (No Condensation) Humidity: 30 - 75% Atmospheric Pressure: 700 - 1,060hPa	
Transportation and Store Environment	Temperature: -10 - 50°C Humidity: 10 - 85% Atmospheric Pressure: 500 - 1,060hPa	

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NSK Europe GmbH
B.A. INTERNATIONAL LTD.