BA OPTIMA


Operation Manual for:

- BA66E
BA611086
- BA25E
BA611083
- BA45E
BA611084


CE 0197 **Made in Japan**

OM-CK600E 001


Please read this operation manual carefully before use, and file for future reference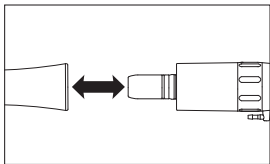


1

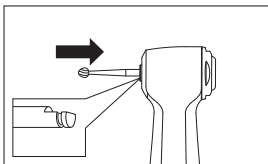
2

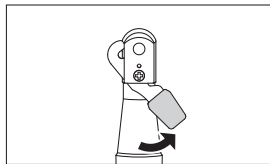
3

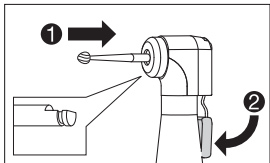
4

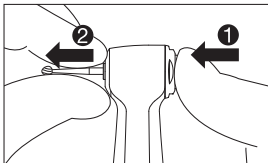
5

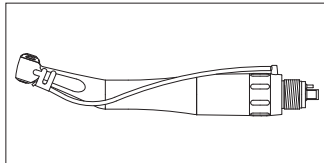6

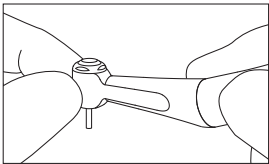7

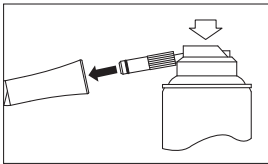8

1. User and Indications for Use

User: Qualified Professionals (Dentist)

Indications for Use:

This medical device is only intended for dental treatment. The device is powered by either an air-motor or electronic micromotor for use in general dentistry. The device is intended for transmitting rotation of the power source at the direct drive ratio or at a different gear ratio, thereby running the instrument such as a bur to cut and polish natural or artificial teeth during dental treatment.

2. Precautions for handling and operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
 WARNING	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.
 CAUTION	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.
NOTICE	General product specification information highlighted to avoid product malfunction and performance reduction.

 **WARNING**

- Immediately after a treatment, the handpiece should be cleaned, lubricated and sterilized. Failure to properly maintain the handpiece may lead to overheating, causing burn injuries or product failure. (Refer to "8. Post-use Maintenance")

- If blood infiltrates inside a handpiece, an automatic handpiece cleaning and lubrication system may not totally clean the internal handpiece components, which may lead to internal blood coagulation. Coagulated blood inside the handpiece may cause handpiece failure and overheating, causing burn injuries. To avoid the risk, B.A. International recommends to immediately lubricate those handpieces with BA ULTIMATE SPRAY.
- Depressing the Push Button while the handpiece is in rotation may lead to overheating, causing burn injuries or product failure. Avoid the push button to contact with any oral tissue. (Except for BA25E)

- Supply coolant water and coolant air while using the handpiece. No supplying the coolant water and coolant air may lead to overheating, causing burn injuries or product failure.
- Do not use the handpiece for polishing purposes. Polishing paste could enter the handpiece, causing the failure of the Push Button or burn injuries due to the overheating of the handpiece head.
- Keep any debris or other foreign materials away from inside the gear or the handpiece. Foreign materials remaining inside may lead to overheating, causing burn injuries or other accidents.

- Ball bearings are wear items and may require replacement. Be sure to inspect the handpiece before use (Refer to "7. Checking the Handpiece Before Each Use"). If abnormal vibration or noise are found, the bearings or other internal parts may be worn out or damaged. Damaged parts may lead to handpiece overheating causing burn injuries. If overheating is suspected, stop using the handpiece immediately and contact your Authorized B.A. International Dealer.

CAUTION

- When operating the product always consider the safety of the patient.
- Users are responsible for the operational control, maintenance and continual inspection of this product.
- Prior to clinical use, inspect the handpiece. Check for vibration, noise and overheating. If any abnormalities are found, stop using the handpiece immediately and contact your Authorized B.A. International Dealer. (Refer to "7. Checking the Handpiece Before Each Use")
- Do not disassemble or alter the handpiece except as recommended by B.A. International in this Operation Manual.
- Do not allow any impact on to the product. Do not drop the product.
- Operators and all others in the area must wear eye protection and a mask when operating this handpiece.

- Do not use burs with problems listed below as the bur may break, seize up or disengage from the chuck.
 - Bent, deformed, worn, rusted, broken, deficient bur.
 - Bur which is cracked on the edge or axis.
 - Non-ISO(EN ISO) standard, or tampered bur.
- Do not use burs other than specified in "16. Specifications" as such burs may be accidentally released or break during rotation.
- Always keep the bur shank clean. Dirt or debris in the chuck could cause poor bur concentricity or low chuck retention force.
- Always insert the bur all the way into the chuck. If insertion is insufficient, premature failure of the bearings or accidental release of the bur may occur.
- Do not exceed Max. Bur Length shown in "16. Specifications".
- Always follow the instructions provided by the bur manufacturer.

- Do not exceed the bur speed recommended by the bur manufacturer.
- Should the handpiece function abnormally during use, stop using the handpiece immediately and contact your Authorized B.A. International Dealer.
- Do not use the following fluids to wipe, immerse or clean the product; strong/super acid water, strong acid/alkaline chemicals, chlorine-containing solutions, solvents such as benzine or thinner.
- The handpiece is delivered in a non-sterile condition and must be autoclaved prior to use.
- Perform regular function and maintenance checks. (Refer to "9. Periodical Maintenance Checks")
- If the handpiece has not been used for a long period, rotate the handpiece and check for noise, vibration and overheating before use.
- To avoid clinical downtime it is recommended that a spare be kept on hand in case of a breakdown during treatment.

- The operation of the handpiece is permitted only on dental units which correspond to the standards IEC 60601-1 (EN 60601-1) and IEC 60601-1-2 (EN 60601-1-2).

3. Accessory List

No.	Part Name	Quantity	Remarks
1	Nozzle for E-fitting	1	-
2	Spray Nozzle	1	BAKIT only

4. Connecting and Disconnecting the Handpiece from the Motor

4-1 Connecting

- 1) Insert the handpiece direct into the motor (Fig. 1).
- 2) Confirm that the handpiece is securely connected to the motor.

4-2 Disconnecting

Hold the motor and the handpiece, then pull apart.

CAUTION

- Do not connect or disconnect the handpiece until the motor has completely stopped.
- Connect ONLY to E type motors (ISO 3964 (EN ISO 3964)).
- Do not exceed the Max. Rotation Speed (Motor) shown in "16. Specifications".

5. Inserting and Removing the Bur

5-1 Inserting the Bur

BA45E / BA66E (Fig. 2)

- 1) Insert the bur until it is correctly secured in place.
- 2) Depress the Push Button and insert the bur into the chuck until the bur "notch" mechanism engages.
Release the button.
- 3) Confirm that the bur is secure by gently pulling and pushing the bur without depressing the Push Button.

BA25E (Fig. 3, 4)

- 1) Turn the Latch Plate to open the chuck.
- 2) Insert the bur into the chuck until the bur securely engages. Swing the Latch Plate back to closed position.
- 3) Ensure that the bur is secure by gently pulling and pushing the bur.

5-2 Removing the Bur

BA45E / BA66E (Fig. 5)

Depress the Push Button to open the chuck (❶) and remove the bur (❷).

BA25E (Fig. 3)

Turn the Latch Plate to open position and pull the bur out.

CAUTION

- Do not use burs with problems listed below as the bur may break, seize up or disengage from the chuck.
 - Bent, deformed, worn, rusted, broken, deficient bur.
 - Bur which is cracked on the edge or axis.
 - Non-ISO(EN ISO) standard, or tampered bur.
- Do not use burs other than specified in "16. Specifications" as such burs may be accidentally released or break during rotation.

- Always keep the bur shank clean. Dirt or debris in the chuck could cause poor bur concentricity or low chuck retention force.
- Always insert the bur all the way into the chuck. If insertion is insufficient, premature failure of the bearings or accidental release of the bur may occur.
- Do not exceed Max. Bur Length shown in "16. Specifications".
- Always follow the instructions provided by the bur manufacturer.
- Do not exceed the bur speed recommended by the bur manufacturer.
- Do not insert or remove the bur until the rotation has completely stopped.
- Do not apply excess pressure to the bur as it may break or bend or become difficult to remove.

6. Connection of the Spray Nozzle

- 1) Attach the Spray Nozzle (Optional) to the handpiece head (Fig. 6).
 - 2) Cut the Irrigation Tube to the desired length then connect to the Spray Tube Nozzle on the motor (Fig. 6).
- *Refer to "13. Option Parts List" to identify the correct Spray Nozzle.

7. Checking the Handpiece Before Each Use

Follow the check procedure below before use. If any abnormalities are found, stop using the handpiece immediately and contact your Authorized B.A. International Dealer.

- 1) Check the Head Cap is firmly tightened.
- 2) Insert the bur. (Refer to "5. Inserting and Removing the Bur")
- 3) Rotate the handpiece for about one minute at the Max. rotation speed of the attached motor. During rotation, check for abnormalities such as abnormal rotation, vibration, noise.
- 4) After the handpiece rotation has completely stopped, touch the handpiece head to confirm the head is NOT heating abnormally (Fig. 7).

 **CAUTION**

- To avoid injury, keep your hands away from the bur during rotation.

8. Post-use Maintenance

After each patient maintain the product as follows.

 **WARNING**

- Immediately after a treatment, the handpiece should be cleaned, lubricated and sterilized. Failure to properly maintain the handpiece may lead to overheating, causing burn injuries or product failure.

 **CAUTION**

- Do not use the following fluids to wipe, immerse or clean the product; strong/super acid water, strong acid/alkaline chemicals, chlorine-containing solutions, solvents such as benzene or thinner.

8-1 Preparation

- 1) Wear a eye protection, a mask and gloves to avoid infection.
- 2) Remove the bur.
- 3) Disconnect the handpiece from the motor.

8-2 Cleaning

■ Manual Cleaning (External Surface)

- 1) Remove dirt and debris on the handpiece surface using a brush under running water (at 38°C or less). Do not use a wire brush.
- 2) Wipe off the moisture on the surface using absorbent cloth.

CAUTION

- If water comes inside the handpiece, blow air into the rear of the handpiece using an air syringe and remove the remaining water.

■ Automatic Cleaning (External and Internal Surfaces)

Use a thermo-disinfector in accordance with ISO 15883 (EN ISO 15883).

Refer to the thermo-disinfector manual.

CAUTION

- After washing with thermo-disinfector and prior to lubrication, dry the product until all internal moisture is thoroughly removed. Thermo-disinfector moisture remaining inside the product could reduce the effect of lubrication and could cause corrosion inside of the product.

8-3 Disinfection

■ Manual Disinfection (External Surface)

Wipe the surface with cloth soaked in disinfectant solution such as alcohol.

■ Automatic Disinfection (External and Internal Surfaces)

Use a thermo-disinfector in accordance with ISO 15883 (EN ISO 15883).

Refer to the thermo-disinfector manual.

8-4 Lubrication

■ BA ULTIMATE SPRAY

Apply BA ULTIMATE SPRAY every time after each use and/or before autoclaving.

- 1) Attach the Spray Nozzle into the Spray Port nozzle on the can.
- 2) Insert the Spray Nozzle into the rear of the handpiece. Hold the handpiece and spray for approximately 2-3 seconds. Apply lubricant until it expels from the handpiece head for at least 2 seconds (Fig. 8).

CAUTION

- Hold the spray can upright.
- Be sure to hold the handpiece firmly to prevent slipping out of hand by the spray pressure when lubricating.
- Supply lubricant until it comes out of the handpiece head.
- If you wish to expel excessive oil from inside of the handpiece, rotate the handpiece for approx. 15 seconds without a bur. During the rotation, do not depress the push button inadvertently, especially when using absorbent cloth to prevent oil from scattering. Depressing the push button during rotation may cause a failure to attach a bur. (except for BA25E)

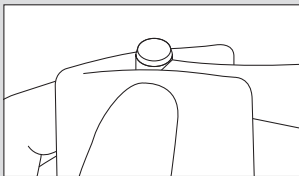
Fig. 9

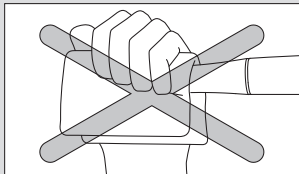
Fig. 10

WARNING

- If blood infiltrates inside a handpiece, an automatic handpiece cleaning and lubrication system may not totally clean the internal handpiece components, which may lead to internal blood coagulation. Coagulated blood inside the handpiece may cause handpiece failure and overheating, causing burn injuries. To avoid the risk, B.A. International recommends to immediately lubricate those handpieces with BA ULTIMATE SPRAY.

8-5 Sterilization

Sterilize the product by autoclave sterilization. After each patient sterilize as noted below.

- 1) Insert into an autoclave pouch. Seal the pouch.
- 2) Autoclavable under the conditions below.
Autoclave for more than 15min. at 132°C, or 3min. at 134°C.
- 3) The handpiece should remain in the autoclave pouch until required for use.

 **CAUTION**

- Be sure to clean and lubricate the handpiece prior to autoclaving. If blood remains on the internal surface it can become clotted and cause product failure.
- Do not autoclave the product with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments.
- Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product.
- To avoid product failure, do not use a sterilizer whose temperature exceeds 138°C during use including the dry cycle. The chamber temperature of some type of sterilizers may exceed 138°C. Contact the sterilizer manufacturer for detailed information.

- Keep the product in suitable atmospheric pressure, temperature, humidity, ventilation, and sunlight. The air should be free from dust, salt and sulphur.
- Do not touch the product immediately after autoclaving as it will be very hot and must remain in a sterile condition.
- Autoclave sterilization is recommended for the product. The validity of other sterilization methods is not confirmed.

NOTICE

- B.A. International recommends Class B sterilizers as stated in EN 13060.

9. Periodical Maintenance Checks

Every 3 months perform periodical maintenance checks, referring to the check sheet below. If any abnormalities are found, contact your Authorized B.A. International Dealer.

Points to check	Details
Rotation	Rotate the handpiece and check for abnormalities such as abnormal rotation, vibration, noise, and overheating.

10. Symbol

This product is Autoclavable up to Max.135°C.

This product can be washed via thermo disinfectant.

Conforms to CE European Directive of "Medical device directive 93/42/EEC."

Manufacturer.

Authorized representative in the European community.

Consult operation instructions.

11. Warranty

B.A. International products are warranted against manufacturing errors and defects in materials. B.A. International reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose or has been tampered with by unqualified personnel or has had non B.A. International parts installed. Replacement parts are available for seven years beyond discontinuation of the model.

12. Spare Parts List

Model	Order Code
Nozzle for E-fitting	NOZE

13. Option Parts List

Model	Order Code	Compatible Product
Spray Nozzle	C1052702	BA45E / BA66E
Spray Nozzle	C032701	BA25E

14. Disposing product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled industrial wastes, to dispose the product for you.

15. Service and Repair

Important

This contra angle handpiece is a result of precision engineering and cannot be dismantled. For service and repair, always send it to either BA International Ltd. or an accredited BA certified service centre without dismantling it.

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NSK Europe GmbH
B.A. INTERNATIONAL LTD.